Instructions for use

SmartSling®

Life Without Limitations®

.....	3
(EN) Instructions for use	5
(DE) Gebrauchsanweisung	7
(FR) Notice d'utilisation	9
(ES) Instrucciones para el uso	11
(IT) Istruzioni per l'uso	13
(NO) Bruksanvisning	15
(DA) Brugsanvisning	17
(SV) Bruksanvisning	19
(FI) Käyttöohjeet	21
(NL) Gebruiksaanwijzing	23
(PT) Instruções de Utilização	25
(JA) 取扱説明書	27
(ZH) 中文说明书	29
(KO) 사용 설명서	31

ENGLISH

WARNING

This sling is supportive only and is not intended or guaranteed to prevent injury. Purchaser agrees to the following:

- Össur accepts no responsibility for any injury which might occur while using Össur products.
- With use of this type of device, it is possible to receive various types of skin irritation, including rashes and pressure points. If you experience any unusual reactions while using this product, consult your health care provider immediately.
- If you have any questions as to how this product applies to your own specific condition, consult your health care provider.
- **If you do not agree to the above conditions, contact your health care provider before using this product.**

GENERAL APPLICATION

- 1 Place the forearm into the SmartSling at a 90° angle, seating the elbow snugly in the corner pocket of the sling. The installed elbow pillow can be removed, if necessary, to improve the fit and comfort.
- 2 Place the thumb through the thumb loop. The thumb loop can be repositioned for an improved fit and increased comfort.
- 3 Close the top flap to secure the sling.
Note: Right arm application is shown in the photo.
For left arm application, the flap will fold towards the body.
- 4 Place the adjustable strap around the neck and fasten it to the sling via the quick-release buckles.
- 5 Position the gusseted chamois neck pad where it is most comfortable.
- 6 If necessary, adjust the shoulder strap from the front (shown here) or back (not shown). If the strap is too long, consult the section on **Trimming Straps**.

IMMOBILIZATION

- 7 Locate the pocket on the side of the sling facing towards the body. Connect the immobilizer strap to the buckle located in the pocket.
- 8 Pull the immobilizer strap around the back and feed it through the loop located on the elbow end of the sling.
- 9 Pull the immobilizer strap back in the opposite direction — on top of itself — and fasten it snugly to itself. If the strap is too long, consult the section on **Trimming Straps**.

ABDUCTION

- 10 Attach one of the included double-sided hook strips to the loop material on the side of the sling facing the body.
- 11 Position the abduction pillow between the body and the sling, using the hook strip installed in step #1 to connect the sling to the pillow.
- 12 If desired, the underarm pillow can be used to provide additional support.
- 13 Pull the abduction pillow strap around the back and attach it to the abduction pillow via the quick-release buckle located on the front of the pillow.
- 14 The abduction pillow strap can be adjusted to obtain the desired fit. If the strap is too long, consult the section on **Trimming Straps**.

EXTERNAL ROTATION

- 1 Using the included double-sided hook strips, attach the wedge pillow to the abduction pillow on the side of the abduction pillow that faces away from the body. Adjust the joined pillows to the desired position. **(Figure 15).**

- 2 Pull the abduction pillow strap around the back and attach it to the abduction pillow via the quick-release buckle located on the front of the pillow (**Figure 13**).
- 3 The abduction pillow strap can be adjusted to obtain the desired fit. If the strap is too long, consult the section on **Trimming Straps (Figure 14)**.
- 4 Attach one of the included double-sided hook strips to the loop material on the side of the sling facing the body (**Figure 10**).
- 5 Secure the sling to the pillow configuration. Adjust the shoulder strap, if necessary, to obtain the desired fit (**Figure 16**).
- 6 If the shoulder strap is too short, it may be replaced with the alternate (longer) strap included in the external rotation kit (**Figure 17**).

TRIMMING STRAPS

All straps on the SmartSling feature alligator clips for easy strap trimming. If any strap is too long, simply remove an alligator clip, trim the strap to the desired length and replace the clip (**Figure 18**).

WASHING & CARE

The liner should be washed by hand using a mild detergent and 30°C (86°F) cold water. Hang to dry.
Do not put the SmartSling in a washing machine or dryer.

LIST OF MATERIALS:

Nylon, Leather, Polyurethane, PVC piping with reflective tape cover, Rubber, Polypropylene, Polyurethane Foam, Polypropylene Foam,

LIMITED WARRANTY

This warranty covers all defects in materials and workmanship or malfunctions of your new SmartSling for 90 days from the date of original purchase.

Össur will repair or replace, at its discretion, any defective or malfunctioning part covered under warranty at no charge. Repairs necessitated by accident, negligence, misuse or abuse by the purchaser are not covered. Also, consequential and incidental damages are not recoverable under this warranty. These limitations may not apply as some states do not allow the exclusion or limitation of incidental or consequential damages.

To obtain service under this warranty, please contact the Inside Sales Department of Össur at (800) 222-4284 to obtain a Return Goods Authorization Number. Once you have obtained the Return Goods Authorization Number, please send your SmartSling (postage prepaid) with a brief written description of the problem to: Össur, 19762 Pauling, Foothill Ranch, CA 92610. **Please write the Return Goods Authorization Number on the outside of the package.** Any product received without a Return Goods Authorization Number will be subject to a 2-week delay.

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Össur Iberia S.A.