



# Underarm Strap Application Instructions

CP020271 Rev A 09/11

This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is not intended to prevent injuries or reduce or eliminate the risk of re-injury.

### INDICATIONS FOR USE WITH ARC 2.0:

For non-surgical or post-operative support and immobilization of the injured shoulder when varying degrees of internal rotation, external rotation and abduction are required. Positions include gunslinger, neutral plane, and statue of liberty.

### INDICATIONS FOR USE WITH ARC 2.0 WITH PILLOW:

For non-surgical or post-operative support and immobilization of the injured shoulder where glenohumeral positioning is key.


**CAUTION:** Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

**CAUTION:** This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.



**CAUTION:** This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150F (65 C.) for any extended period of time. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

**WARRANTY:** This device is warranted to be free from defects in material and workmanship for a period of 90 days. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at [www.BledsoeBrace.com](http://www.BledsoeBrace.com), or by calling 1-888-BLED SOE (253- 3763).

## APPLYING THE UNDERARM STRAP TO THE ARC 2.0

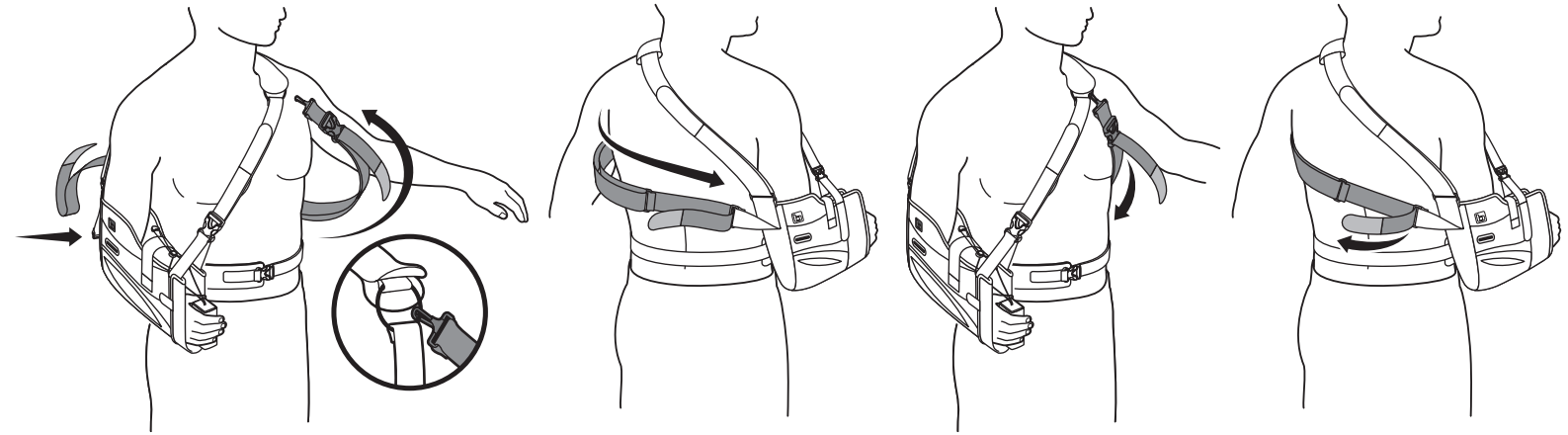


## APPLYING THE UNDERARM STRAP TO THE ARC 2.0 WITH PILLOW

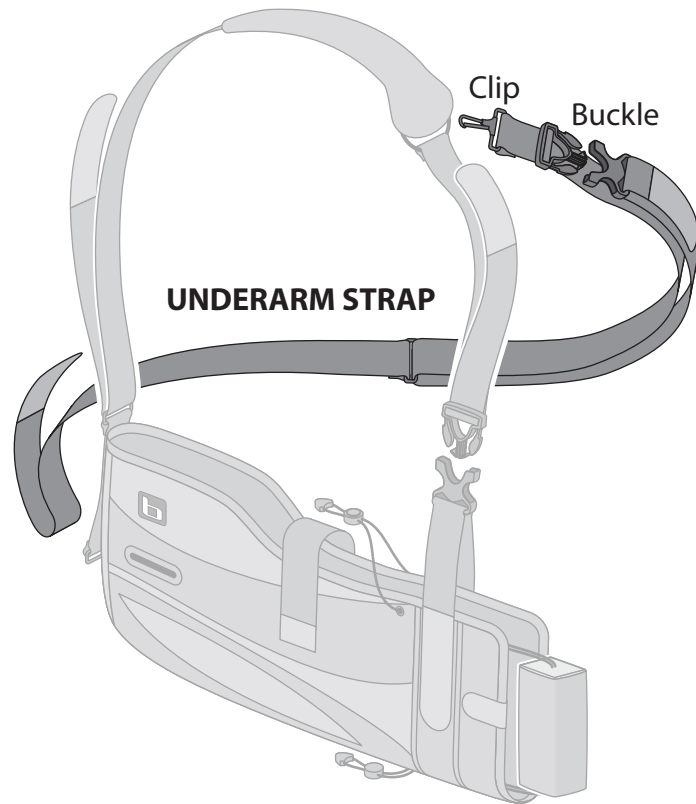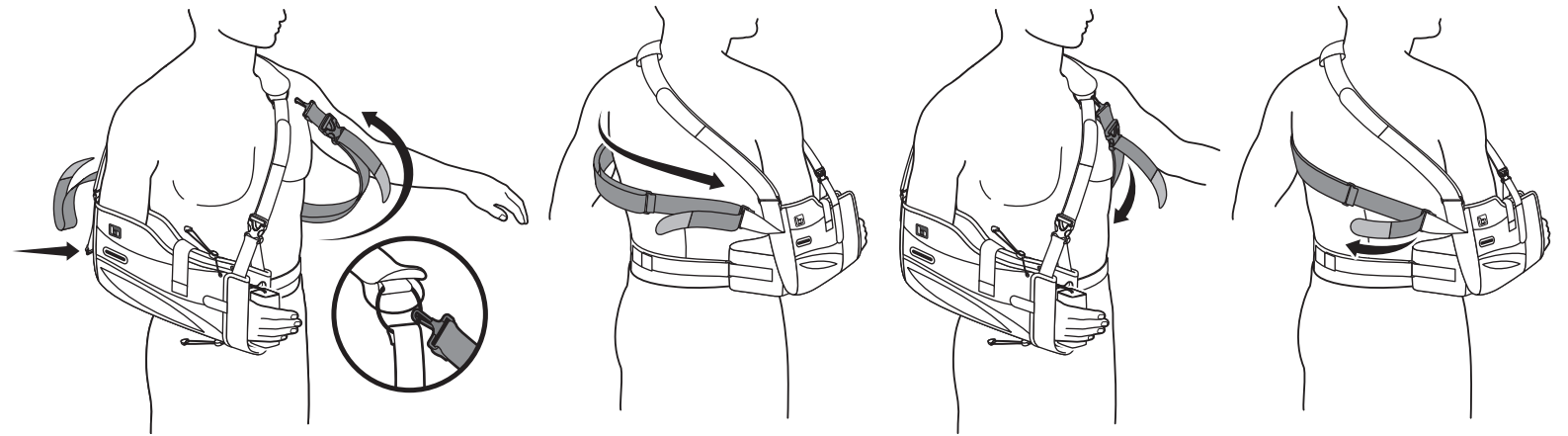

### EXAMPLE: RIGHT SHOULDER

Attach clip to the ring inside neck pad and pass underarm strap under the unaffected shoulder. Attach loose end of the underarm strap to the D-ring at the back end of the sling. Loosen the hook closures at either end of the strap and pull the strap ends, front and back, to a comfortable tension. Refasten the closures.

NOTE : Excess strap may be trimmed if necessary.

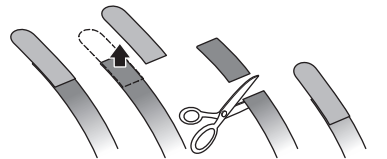

For product information or questions pertaining to sales or service, please contact Bledsoe Brace Systems™.

EUROPEAN AUTHORIZED REPRESENTATIVE



© 2011 Medical Technology Inc., d/b/a Bledsoe Brace Systems  
All Rights Reserved.  
2601 Pinewood Drive, Grand Prairie, Texas, 75051, USA  
Toll Free Tel.1.888.BLED SOE (253.3763)  
1.800.527.3666 • Local 972.647.0884  
Local FAX 972.660.5495 • International FAX 972.606.0649  
[www.BledsoeBrace.com](http://www.BledsoeBrace.com)

Emergo Europe  
Molenstraat 15  
2513 BH The Hague  
The Netherlands  
Tel: (+31) 70 345 8570  
Fax: (+31) 70 346 7299