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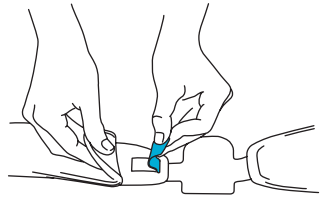
CARE

The entire Pediatric Ankle Brace can be hand washed in luke warm water with mild soap and air dried.

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LATEX

All Aircast products are latex-free.

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WARRANTY POLICY

Satisfaction – Aircast will provide prompt refund for any product that does not satisfy the physician for any reason whatsoever.

Durability – The Pediatric Ankle Brace is designed to last for as long as required by the original patient. Aircast may, at its discretion, furnish replacement parts during this time, provided the defective part is returned to Aircast for analysis.

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REFERENCES

1. Hamilton WG: Sprained Ankles in Ballet Dancers. *Foot and Ankle* 3-2: 99-102, 1982

US PATENTS: 3,955,565 4,280,489 4,628,945 5,125,400 5,113,877 5,492,133 AND OTHER PATENTS PENDING

Air-Stirrup® Ankle Brace

pediatric
pédiatrique
para pediatría
(02J)

AIRCAST^{LLC}

92 River Road
Summit, NJ/USA 07902-0709
(908) 273-6349

800-526-8785

Fax (800) 457-4221

Fax (908) 273-1060

R 12/20/04
02J100F
0224

AIRCAST^{LLC}

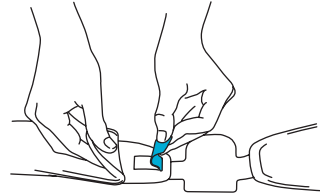
Air-Stirrup® Ankle Braces apply graduated, pulsating compression and are “suitable at some stage for virtually every ankle injury”.¹

With each step, the compression of the conformable aircells milks away edema and swelling. When worn with a lace shoe, the Air-Stirrup provides support to prevent further injury by effectively limiting inversion and eversion.


Instructions:


1 If necessary, lift bottom section of aircells until the heel straps are fully exposed. Peel up heel straps and adjust heelpad width. Firmly press heel straps then aircells back in place.

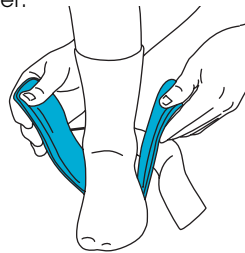2 Put on an absorbent sock. Position heel on heelpad with round edge of heelpad facing heel. Align brace sides with ankle. Secure strap. Put on lace shoe or sneaker.

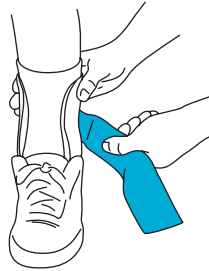3 Squeeze brace sides together with one hand while tightening strap with the other hand. Tighten to a comfortable compression.

4 To remove, disconnect strap from inner brace side, leaving strap attached on outer brace side. This will preserve the correct fit. To reapply brace, position heel on the heelpad and secure strap onto inner brace side.

The Pediatric Ankle Brace aircells are preinflated and **do not** require adjustment.

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