

Instructions:


- Put on sock
- Unfasten strap from black plastic ring, open top flaps, and open brace
- Slide foot, toe first, into top of brace
- Pull brace stabilizers upward and align stabilizers with sides of ankle

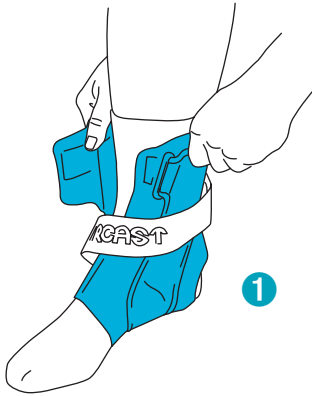- Wrap flaps around top of ankle and secure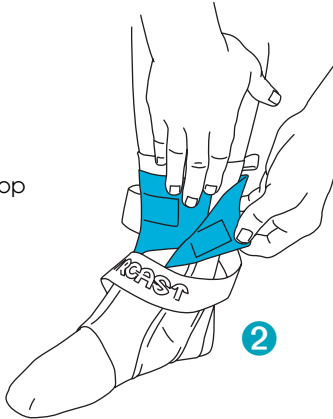

- Insert end of strap into black plastic ring
- Bend knee and align over toes
- Pull strap until snug and secure strap end onto body of strap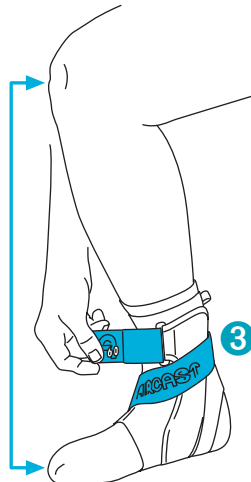

To adjust fit, tighten or loosen strap

MAINTENANCE

Designed for single person use only.

INDICATIONS FOR USE

The Aircast® A60™ Ankle Support is designed to provide support and protection for ankles.

LATEX

This product is latex-free.

CARE

The entire A60 Ankle Support can be hand washed in lukewarm water with mild soap and air dried.

WARRANTY POLICY — A60

Aircast will provide prompt refund for any product defect for a period of 90 days from purchase.

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"A60" and "The Next Degree in Ankle Support" are trademarks of Aircast LLC, Summit, New Jersey, USA.

"Breath-O-Prene" is a registered trademark of AccuMED Technologies Inc., Buffalo, New York, USA.

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"A60" y "The Next Degree in Ankle Support" son marcas de Aircast LLC, Summit, New Jersey, EE.UU.

"Breath-O-Prene" es una marca registrada de AccuMED Technologies Inc., Buffalo, New York, EE.UU.

EU Representative

AIRCST
EUROPE GmbH

Georg-Wiesböck-Ring 12
D-83115 Neubuern
Tel: 00800-00 85 24 33 (free phone)
0049 8035 87 99 50
Fax: 00800-00 85 40 15 (free fax)
0049 8035 87 99 87

Manufacturer

CE AIRCAST
LLC

92 River Road
Summit, NJ/USA 07902-0709
Tel: (800) 526-8785
Tel: (908) 273-6349
Fax: (800) 457-4221
Fax: (908) 273-1060

PATENT PENDING

R 12/23/05
02T105C
0218I

Chevillère
Tobillera

(02T)

The Next Degree
in Ankle Support™

AIRCST
LLC

AIRCAST[™]

A60™ ANKLE SUPPORT

The Next Degree in Ankle Support.™

The A60 uses a sleek stabilizer molded at a 60 degree angle to help prevent ankle sprains. Located on each side of the ankle, the stabilizer guards against ankle rollover by securely holding the ankle in place. Patented Breath-O-Prene® fabric keeps the A60 cool and comfortable while the anatomical design fits easily in athletic footwear without adding bulk. A single strap is used for both application and adjustment — eliminating time-consuming lacing or taping.

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