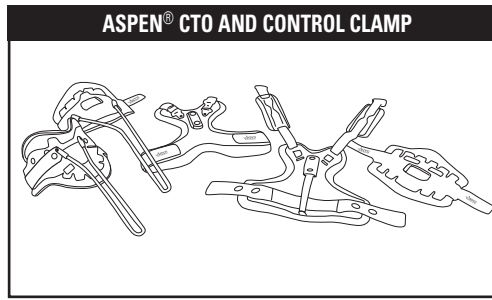
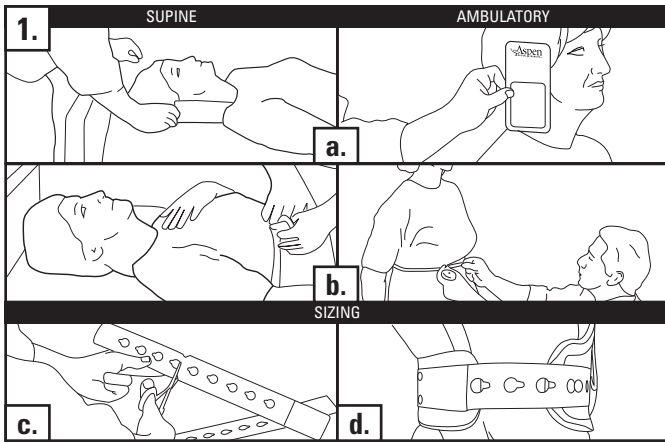
**SUPINE APPLICATION (2-POST) and
AMBULATORY (4-POST) APPLICATION**

Aspen Medical Products, Inc.
6481 Oak Canyon, Irvine, CA 92618
800-295-2776 • 949-681-0200
www.aspenmp.com

ENGLISH
SIZING

a. Use the Aspen sizing guide to measure the patient for the correct size Aspen Collar. The sizing guide can be found at www.aspenmp.com.

b. Measure the chest circumference 1 inch below the tip of the Xiphoid process.

c. For circumferences smaller than 39", trim the ends of the belt panels at the notch.

d. Attach the belt by aligning the keyhole over the aluminum rivet and pull to lock in place. Do this on both sides.

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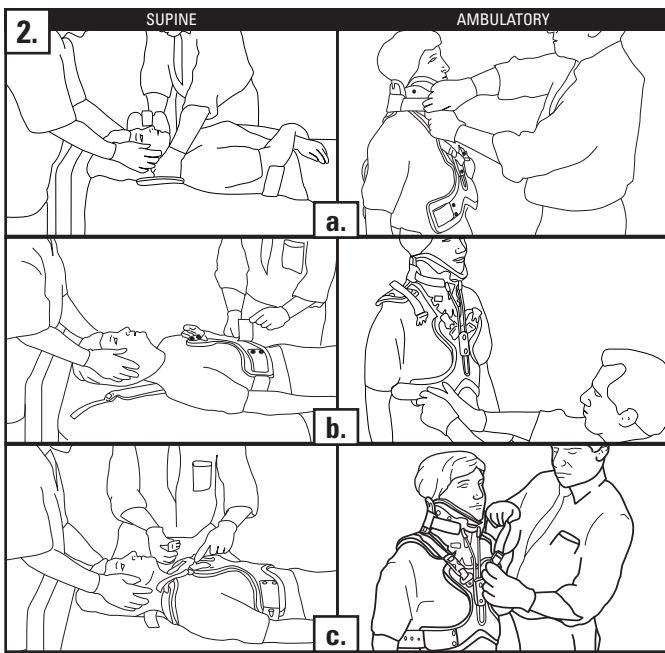
ENGLISH
FITTING

a. Fit the collar to the patient as described in the Aspen collar instruction sheet. The instruction sheet can be found at www.aspenmp.com.

b. (Supine: Slide the CTO underneath the patient while compressing the mattress cover. Logroll the patient if needed.) Line up the control clamp to the centerline on the chest and attach the Velcro loop.

c. Snap the shoulder straps into place and loosely tighten. Bars may be bent to accommodate patient anatomy.

Note: To relieve pressure at the front on the collar, bend bars away from the patient.

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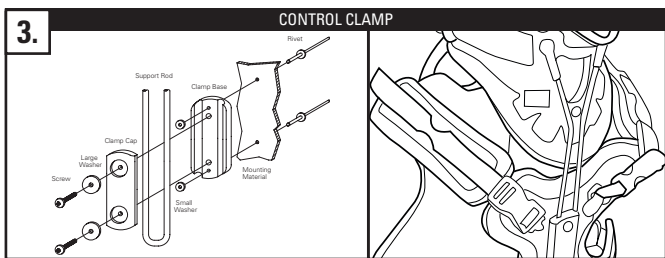
ENGLISH
CONTROL CLAMP

Use a phillips head screw driver to loosen screws to adjust the height of the bars. Tighten the clamp after adjustment.

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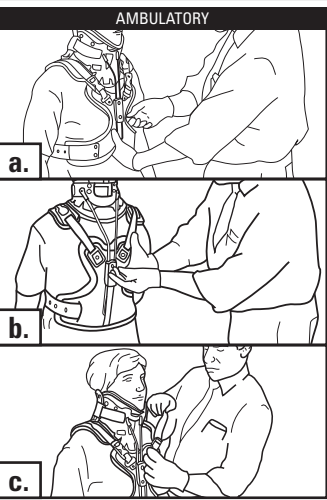
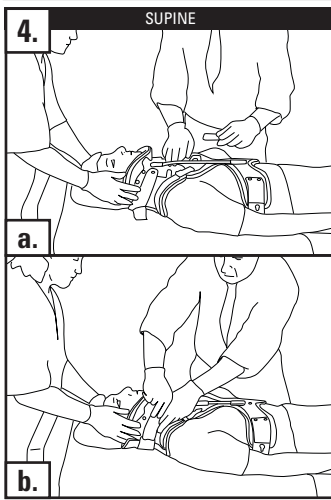
ENGLISH
ADJUSTMENT

a. Slide the support posts on the front of the CTO upward until the patient's head is properly supported. Tighten the screws on the support clamp.

b. Adjust the collar back in the same manor. (Hint: the higher the back of the collar, the more effective the motion restriction.)

c. Check for symmetry and tighten shoulder straps removing slack.

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ENGLISH
NOTES

- The Aspen® CTO can be altered to mirror changes in the patient's condition. For supine patients where posterior support posts are desirable, a pillow may be used under one shoulder to minimize pressure caused by the posterior support post.
- Be aware that excessive tightening of the shoulder straps will cause traction.
- Do not use a full size screwdriver to tighten or adjust the support post clamps while the device is on the patient.
- When advancing to a 4-post configuration, ensure that the post strap covers the support post.
- For kyphotic patients or for others with postural abnormalities, it is possible to bend the support posts for a better fit. To accomplish this, use a pair of bending irons or remove the posts from the clamp and bend using a vise to ensure that they remain symmetrical. Be careful not to weaken the posts by bending them repeatedly or by bending them more than 45 degrees.
- Replacement pads are available from your doctor, local orthotic prosthetic facility, or by calling Aspen Medical Products at 800-295-2776.

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LIMITED WARRANTY
Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or parts(s). Aspen Medical Products shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS. This warranty gives you specific legal rights and you may have other rights which vary from state to state. Warning: To ensure proper use, please review all material. A listing of authorized distributors in your area is available upon request.

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.