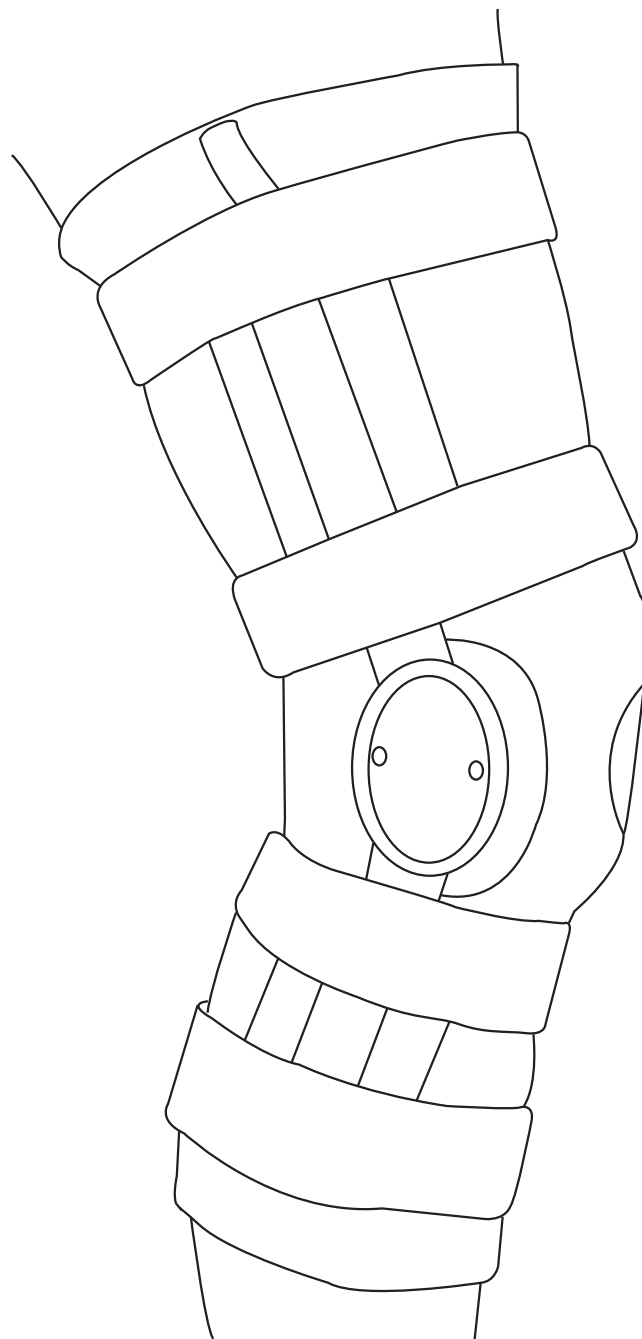
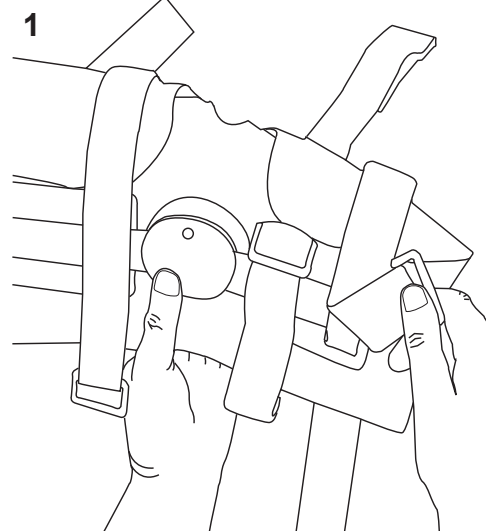
Crossover

Crossover • Crossover Short • Crossover ROM
Application Instructions CP020131 Rev H 01/11

Crossover • Crossover Short • Crossover ROM Front Closure Versions

1. Loosen all straps.
Unfasten all the straps and detach from one upright, leaving the D-ring end attached to the other upright. This allows more freedom when fitting the brace to the leg.

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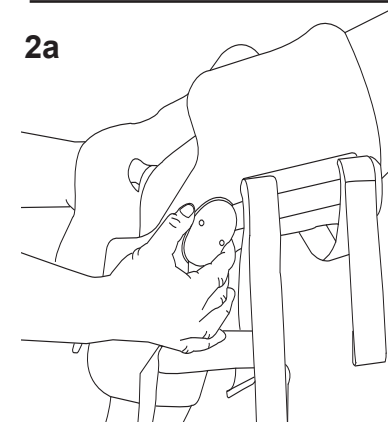
2a

2. Apply Spacer Mesh Wrap.
a. While seated with the leg bent at 60°, and without removing the hinges or the straps, wrap the sleeve around the leg so that the hinges are lined up with the center of the knee.
b. Close the lower wrap with the hook-to-pile fastener.

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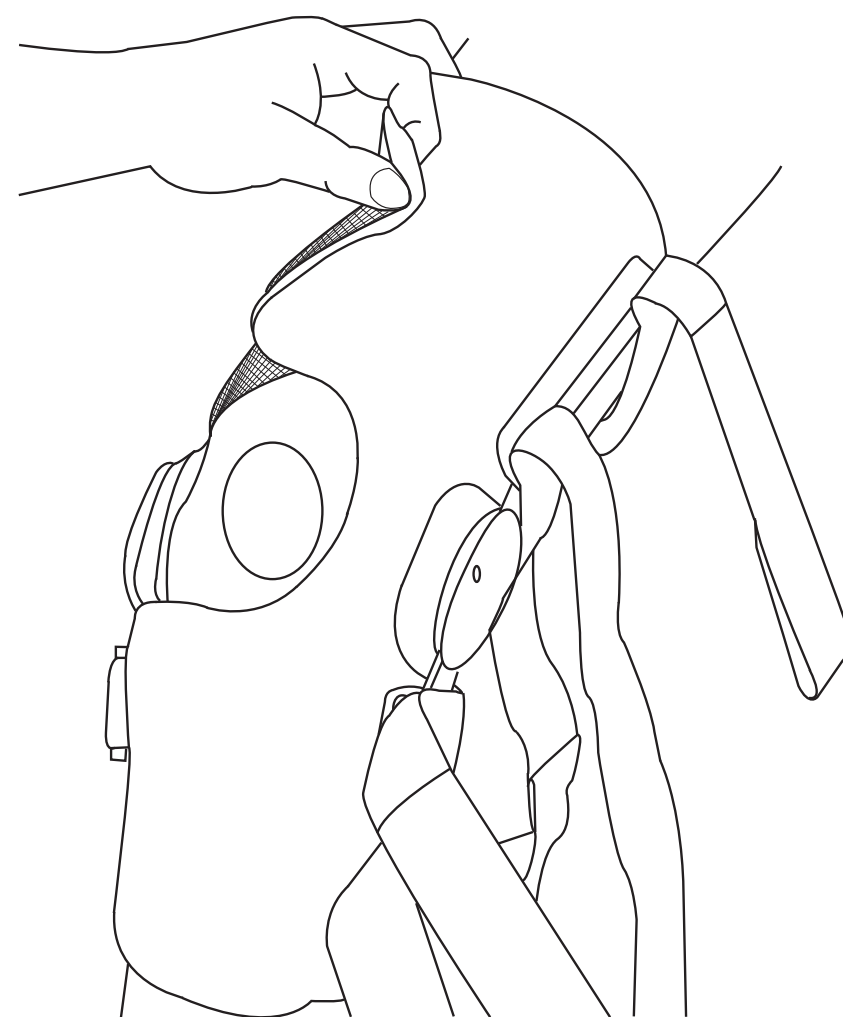
3

3. Close Upper Wrap.
Lightly position the upper hook-to-pile fastener. If necessary, re-position the wrap for an even fit. The front hole must be positioned directly over the knee cap (patella) as shown above.
NOTE: Hinge uprights should be checked for alignment prior to securing the straps.

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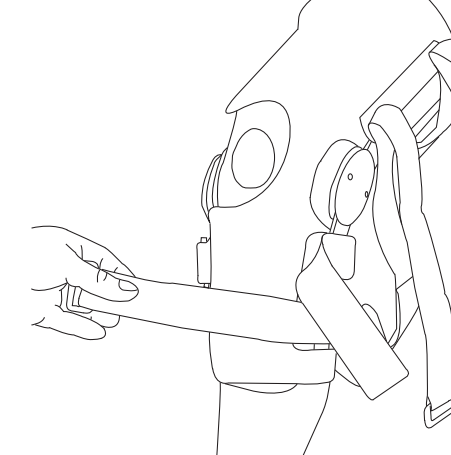
4

4. Secure Lower Strap.
Wrap the lower strap around the brace, then loop the strap through the D-ring. Pull back to tighten, and press to close.

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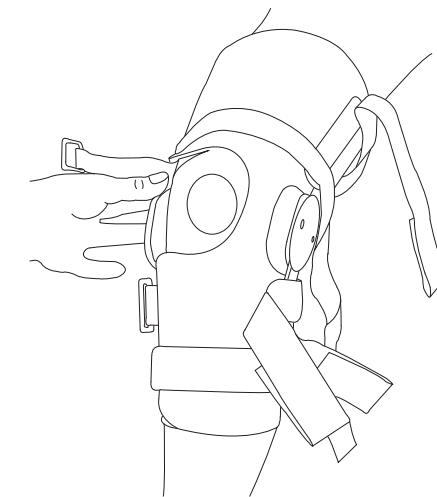
5

5. Secure Upper Strap.
Wrap the upper strap around the brace, then loop the strap through the D-ring. Pull back to tighten, and press to close. Secure the two remaining straps in the same manner.
NOTE: The Crossover Short has only two straps.

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
Crossover ROM Hinge Instructions

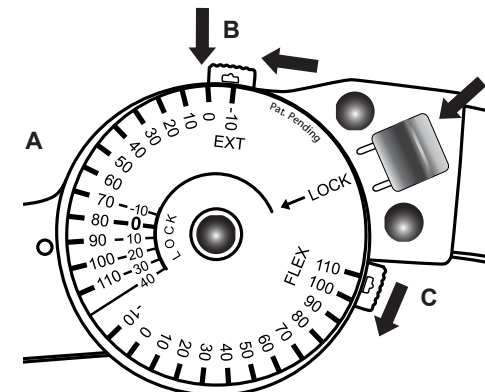A. Move uprights to the desired position between the prescribed range of motion (ROM).
B. Press the gray extension button and slide it to the desired setting. The button will spring out to set the extension range of motion.
C. Press the gray flexion button and slide it to the desired setting. The button will spring out to set the flexion range of motion.
D. To lock the brace at -10, 0, 10, 20, 30, or 40 degrees, move the uprights aligning the indicator mark with the prescribed setting in orange. Then slide the orange lock tab toward the hinge. This will lock the brace in place.

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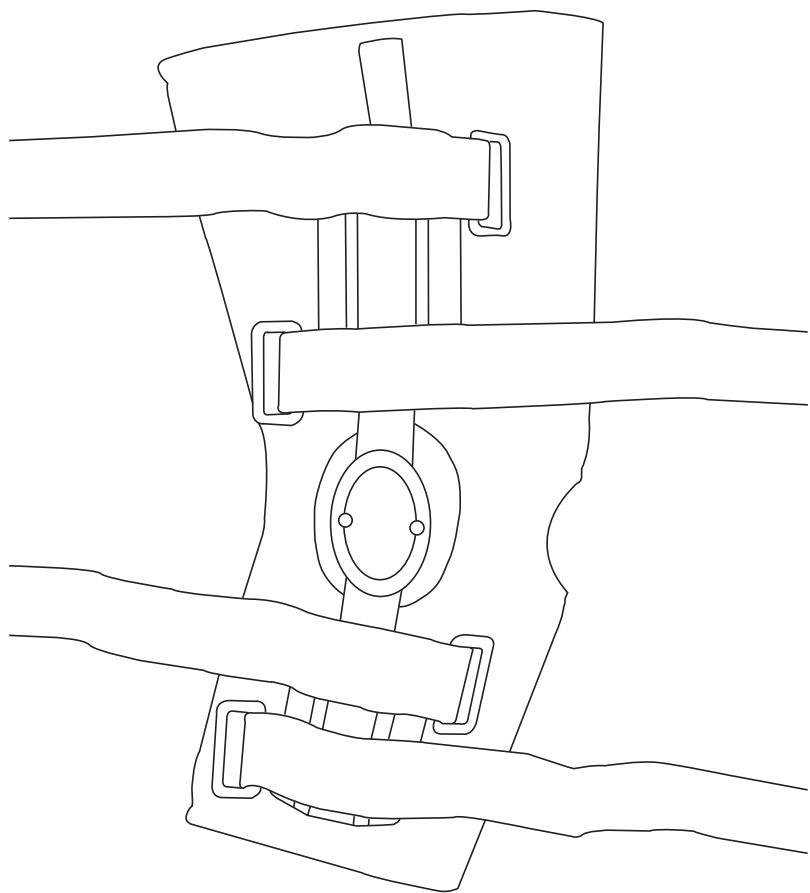
Crossover • Crossover Short Pull-Up Versions

1

1. Loosen All Straps.
Loosen all straps to their maximum circumference, leaving D-ring end attached to hinge arm. This allows more freedom when pulling the brace onto leg.

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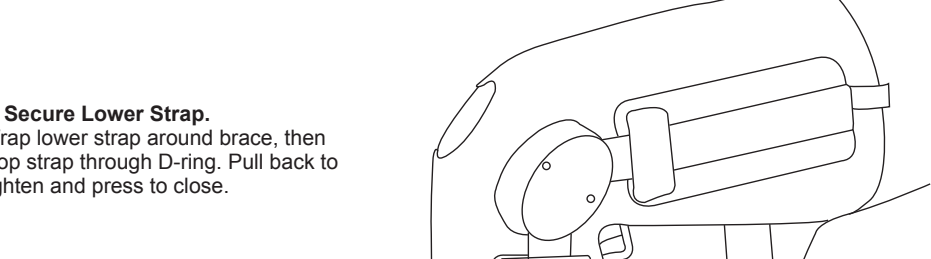
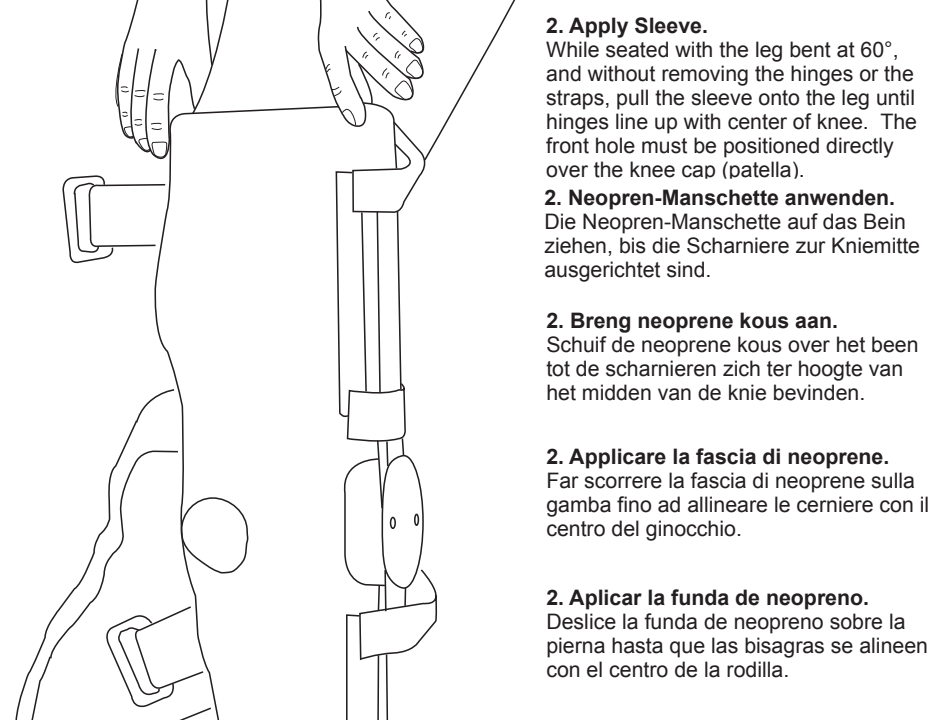
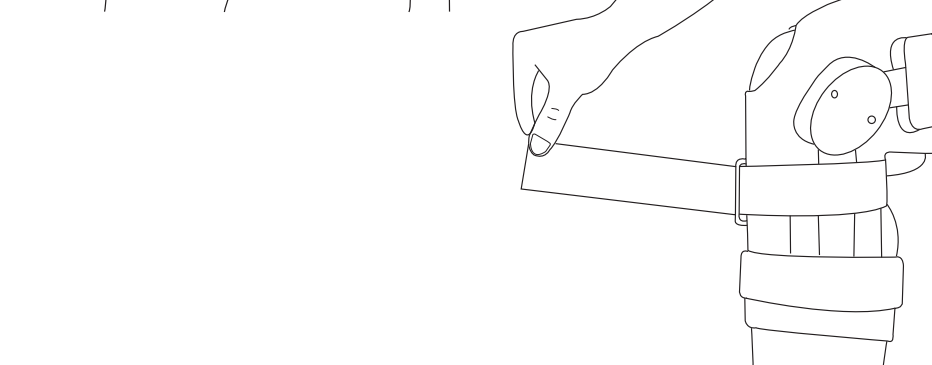
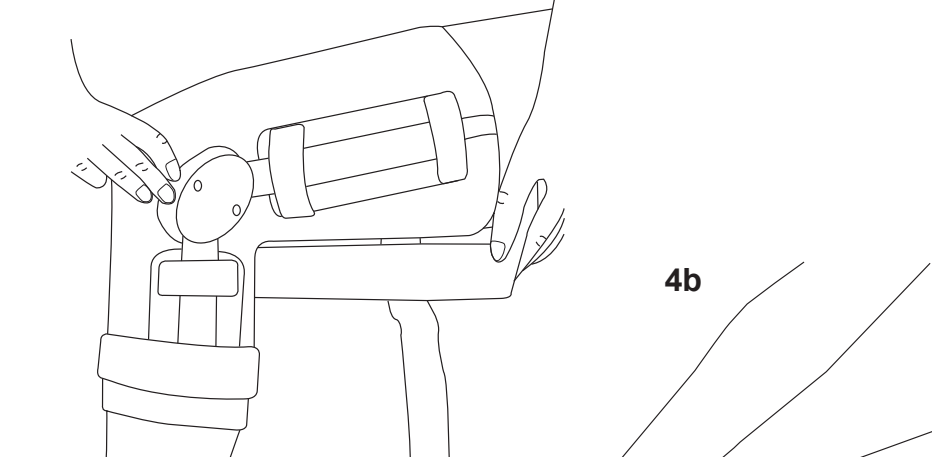
4a

4. Secure Top Straps
a. Wrap strap around brace, then loop strap through D-ring.
b. Pull back to tighten, press to close.
NOTE: Crossover Short has two straps and the Crossover Brace has four straps (Crossover Brace is shown).
c. On Crossover Brace, repeat steps on straps three and four, alternating front to back.

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This device is offered for sale by or on the order of a physician or other qualified medical professional.

Indications: Indicated for immediate application following injury or operative procedures to reduce pain and swelling and aid in healing.

Caution: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

Caution: The range-of-motion stops on both the medial and lateral hinges must be set the same.

Caution: This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.

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For product information or questions pertaining to sales or service, please contact the national distributor in your area or Bledsoe Brace Systems directly.

Manufactured by:
Bledsoe Brace Systems™
2601 Pinewood, Grand Prairie, Texas, 75051, USA
Toll Free Tel. **1.888.BLED SOE (253.3763)**
Or 1.800.527.3666 • Local 972.647.0884
Local FAX 972.660.5495
www.BledsoeBrace.com

Caution: This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents, or exposure of the hinges to debris or liquids. Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65°C) for any extended period of time. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

Warranty: This device is warranted to be free from defects in material and workmanship for a period of 90 days. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at www.BledsoeBrace.com, or by calling 1-888-BLED SOE (253-3763).

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