


LumboLux


APPLICATION INSTRUCTIONS

CP020514 Rev A 07/10 L


This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient. This device is not intended to prevent injuries or reduce or eliminate the risk of injury.

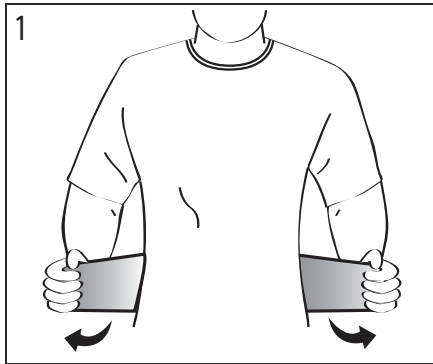


1. Position the support against lower back.

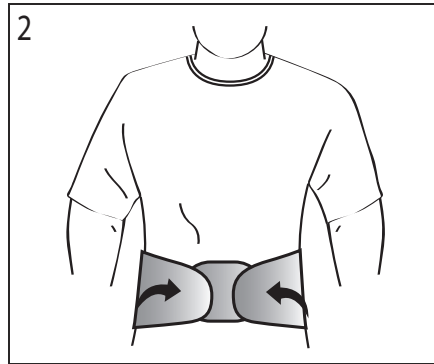

2. Wrap support around waist.

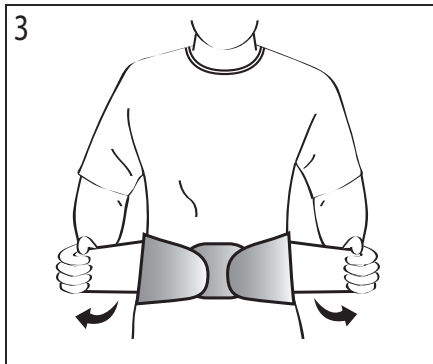

3. Pull and attach lumbar support.

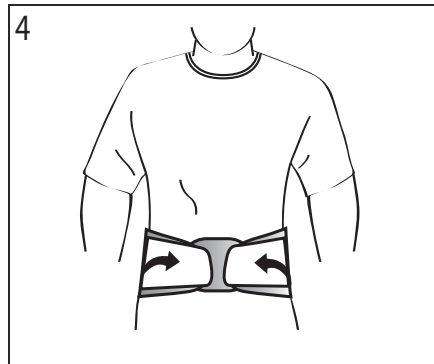

INDICATIONS: Indicated for chronic back pain, discogenic disc disease, lumbar muscle or ligament sprain or strain, spondylolithesis, spondylolysis, and when disc pressure unloading is desired.

WARNINGS: Do not leave this device in a car on a hot day or in any other place where the temperature may exceed 150°F (65°C) for any extended period of time. If any additional pain or symptoms occur while using this brace, discontinue use and seek medical attention.

WARRANTY: This device is warranted to be free from defects in material and workmanship for a period of 90 days. This warranty applies to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at www.BledsoeBrace.com, or by calling 1.888.BLEDSOE (253.3763).