


Padded PF Splint


APPLICATION INSTRUCTIONS

CP020433 Rev A 08/08


This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient. This device is not intended to prevent injuries or reduce or eliminate the risk of injury.

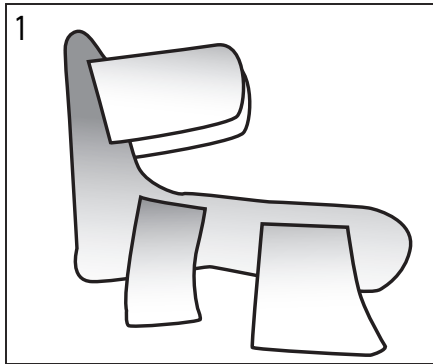


1. Unfasten Straps
Unfasten each of the three straps.

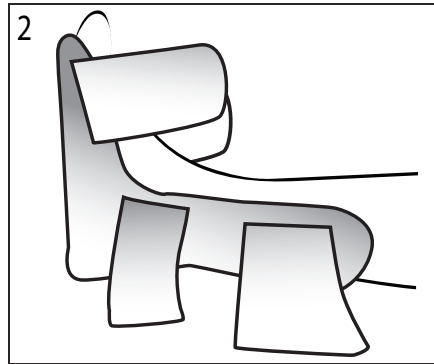

2. Position the Foot
Position the foot inside the splint with the foot flexed to provide the maximum angle of dorsiflexion.

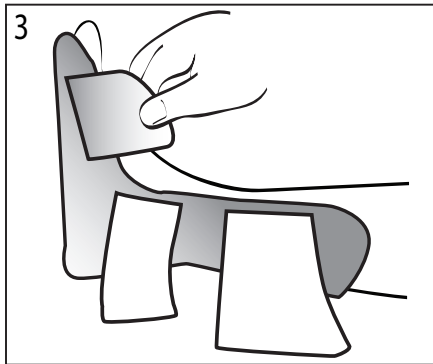

3. Secure Forefoot Strap
Secure the forefoot strap using the fastener.

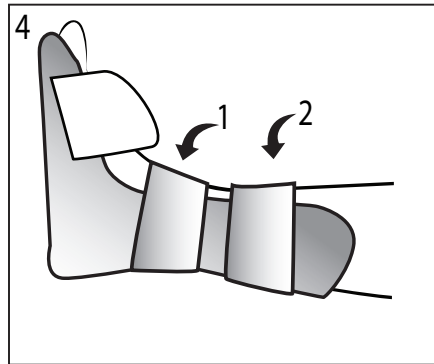

4. Secure Remaining Straps
Beginning with the ankle strap, secure the remaining straps using the fasteners.

INDICATIONS: Indicated for plantar fasciitis to aid in stretching the Achilles Tendon, maintaining the foot in dorsi-flexed position, and maintaining the length and/or position of the plantar fascia in nonweight-bearing conditions.

CONTRAINDICATIONS: Contraindicated for use by neuropathic diabetic patients with ulcerations on the plantar surface of the foot.

WARNINGS: Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150°F (65°C) for any extended period of time. If any additional pain or symptoms occur while using this brace, discontinue use and seek medical attention.

WARRANTY: This device is warranted to be free from defects in material and workmanship for a period of 90 days. This warranty applies to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at www.BledsoeBrace.com, or by calling 1.888.BLEDSOE (253.3763).



Council Directive 93/42/EEC
of 14 June 1993 concerning Medical devices

MANUFACTURED BY:
BLEDSOE BRACE SYSTEMS™
2601 Pinewood; Grand Prairie, Texas, 75051, USA
Toll Free Tel: 1.888.BLEDSOE (253.3763) • 1.800.527.3666
Local: 972.647.0884 • Local Fax: 972.660.5495
International Fax: 972.606.0649
www.BledsoeBrace.com

EUROPEAN AUTHORIZED REPRESENTATIVE



EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands
Tel: (+31) 70 345 8570
Fax: (+31) 70 346 7299