



# Clavicle Support

## FITTING INSTRUCTIONS

**BREG**  
an Orthofix Company  
**BREG Inc.**  
Vista, CA 92081 U.S.A.  
**WWW.BREG.COM**

Telephone (800) 321-0607  
(760) 599-3000  
Telefax (760) 598-6193

AW-1.90213 REV A

**W A R N I N G S**

**CAUTION:** FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.


**CAUTION:** FOR SINGLE PATIENT USE ONLY.

**CAUTION:** IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.


**WARNING:** CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE.

**WARNING:** THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.


**1** Unfasten hook and loop closure.

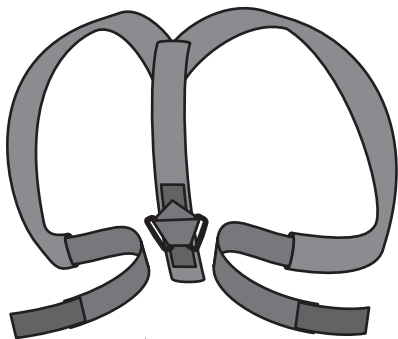



**2** Position Clavicle Brace such that the center of the brace is centered on the patient's back. The D-rings should rest furthest from the patient's neck.

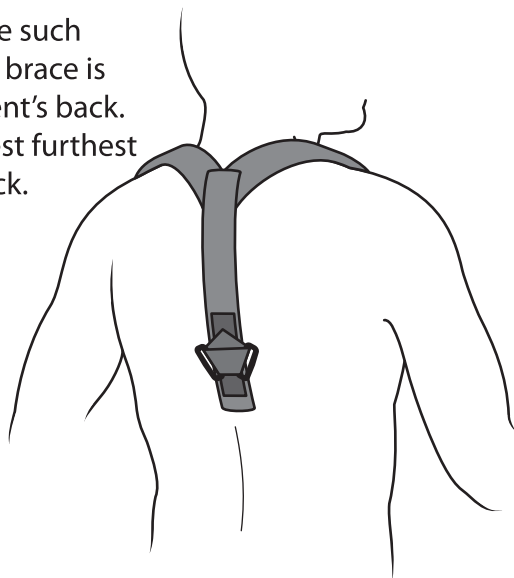


**3** Position straps around the top of the patient's shoulders and wrap under the arm pits. Thread the side straps through the D-rings.

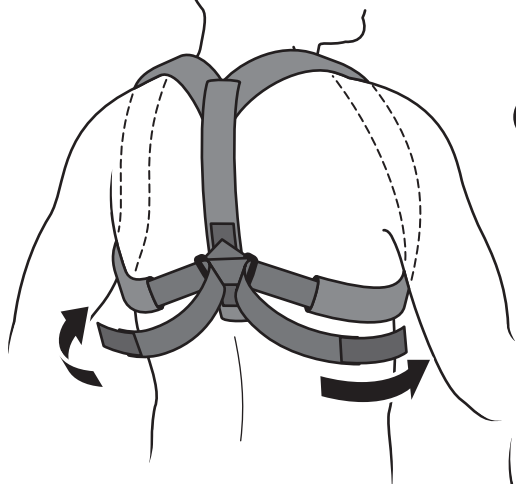

**4** Tighten the straps, to pull the shoulders back, and fasten the hook tabs on the ends of the straps to the brace.

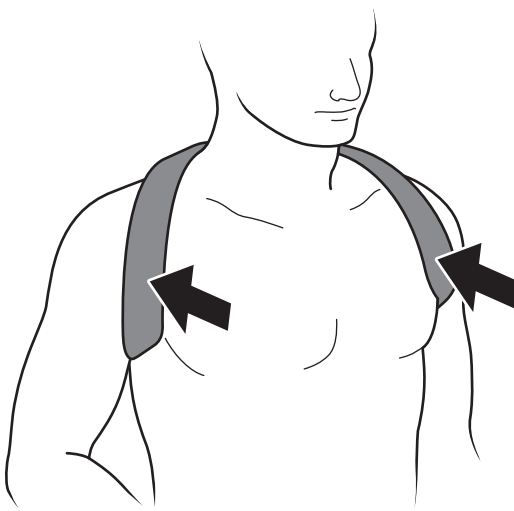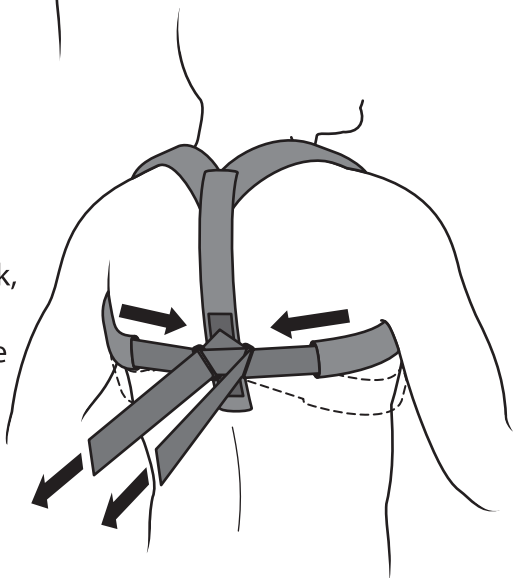

**5** The D-ring attachment site can be repositioned on the center strap for additional support and comfort.

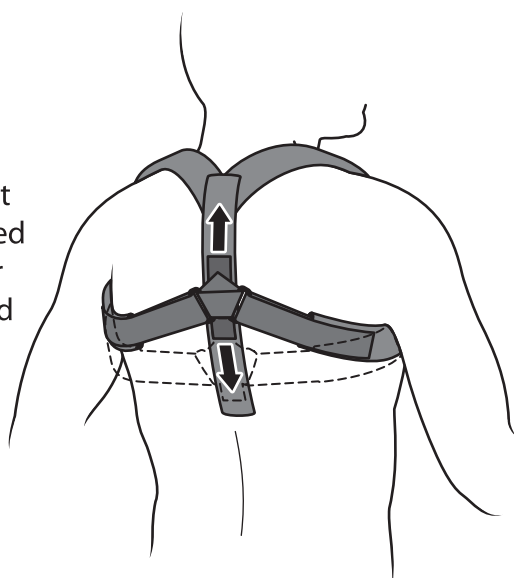

**Washing Instructions:** Fasten hook and loop straps. Hand wash in warm water using a mild detergent. Air dry.