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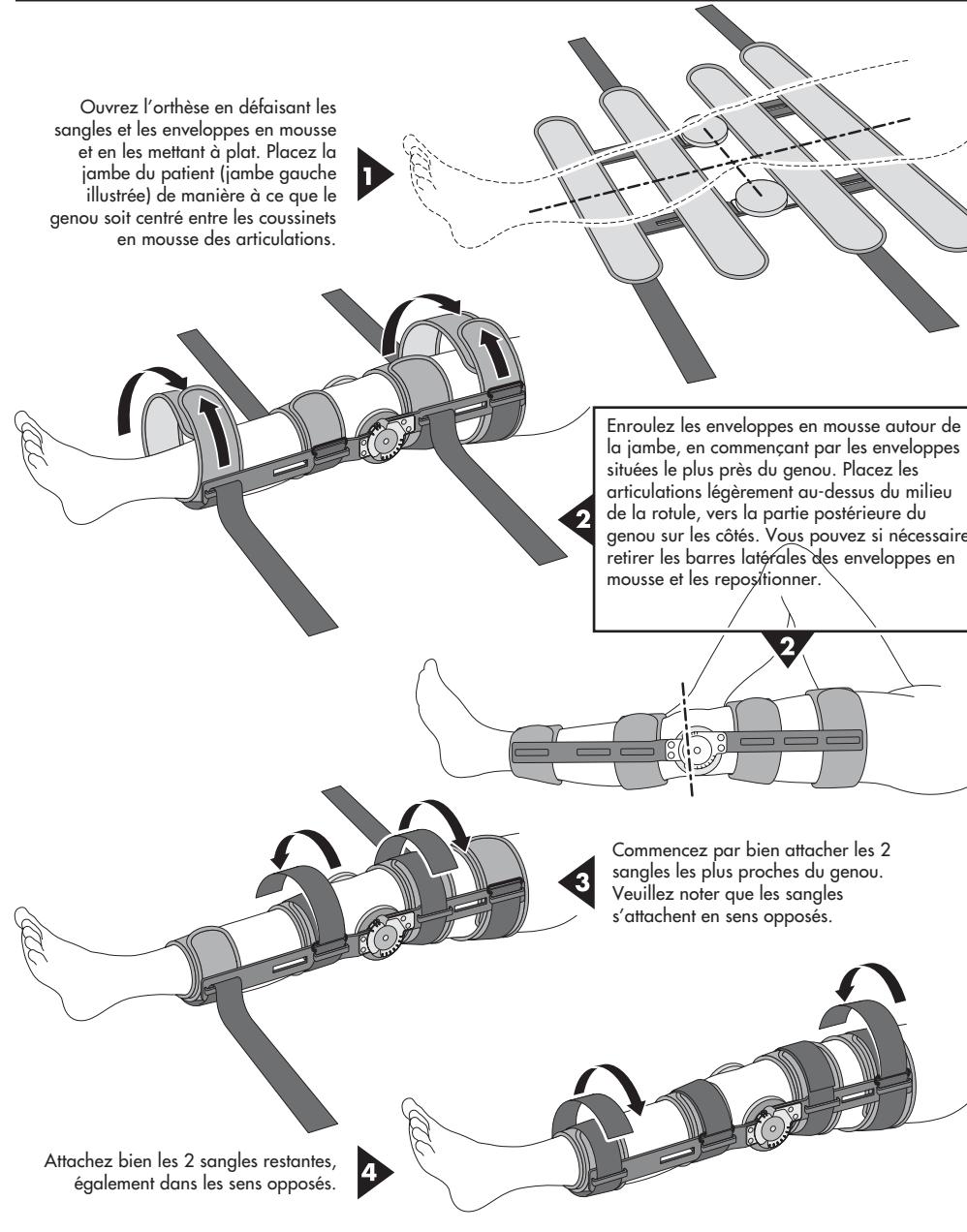
Telefax (760) 598-6193

PN-1.04570 REV E 6/03

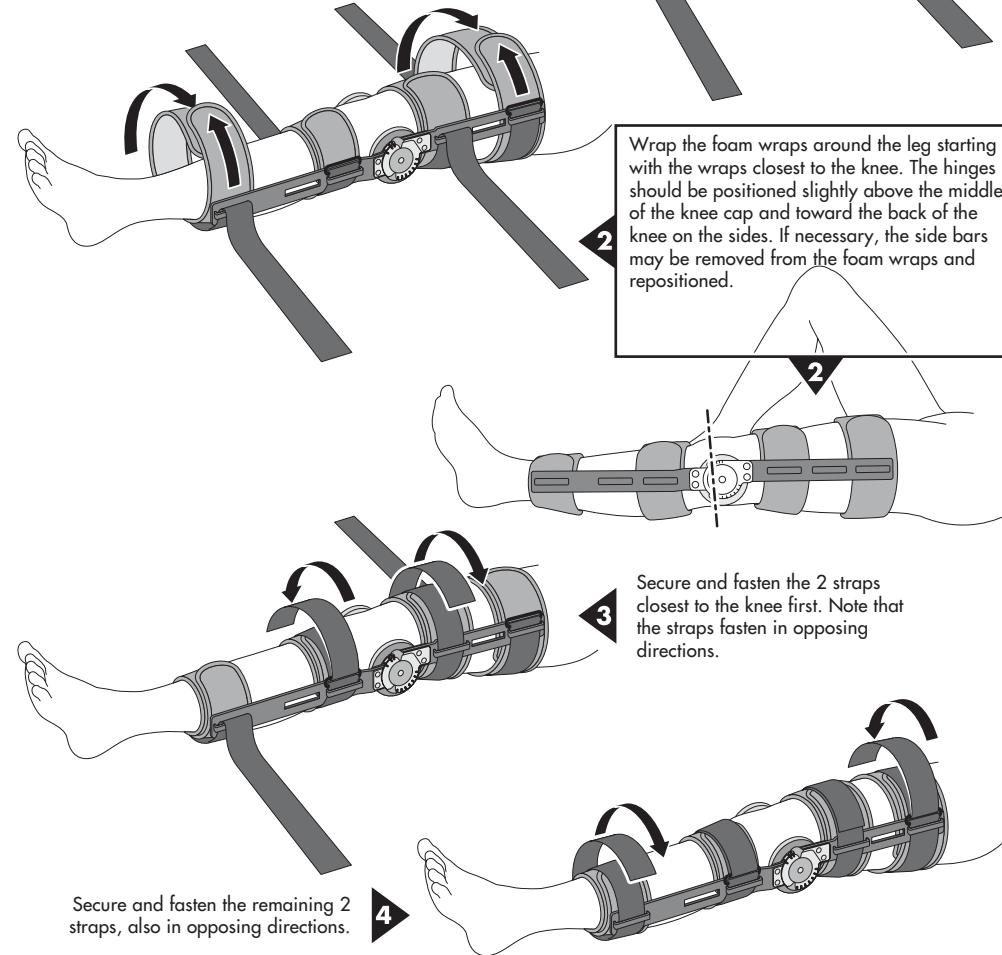
EC responsible person
93/42/EEC Annex 1 Essential Requirements
MDSS
Hauptstrasse 39
D-30974 Wennigsen, Germany

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.
CAUTION: FOR SINGLE PATIENT USE ONLY.
CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
WARNING: CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.
WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM


POSTOP LITE KNEE BRACE

Open the brace by releasing the straps and foam wraps and laying them out flat. Position the patient's leg (left leg illustrated) so that the knee is centered between the foam hinge pads.


TIPS ON USE AND CARE OF YOUR POST-OP LITE KNEE BRACE:

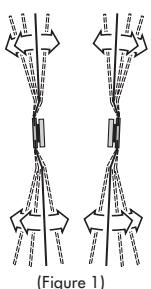- Close the foam wraps in the front to allow easy removal of the brace later.
- The foam wraps may be trimmed with scissors as needed.
- The side bars may be bent as illustrated in Figure 1 to accommodate individual legs. Bend in equal amounts above and below the hinge and on each side.
- Wash foam wraps and straps by hand only and air dry. Extra wraps are available from BREG.
- To tighten straps securely it is easiest to use two hands—one to pull the end after it passes through the ring and the other to push the strap on the leg toward the ring.

QUICK-SET™ HINGE ADJUSTMENT INSTRUCTIONS:

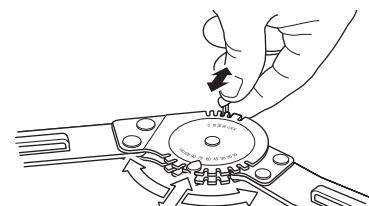To adjust the hinge, simply pull the stop out of its slot and move it to the appropriate slot that corresponds to the degree of flexion or extension desired. The slot identified as "LOCK" is used for locking the brace in full extension.

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