

ADHESIVE VASCULAR ACCESS DEVICE DRESSING

# COVADERM PLUS<sup>®</sup> VAD

## COVADERM PLUS<sup>®</sup> VAD



- Ideal bacterial barrier for I.V. sites, central lines and dialysis catheters
- Replace gauze, tape and scissors with an easier solution
- Rounded corners make a difference
- Easy to use



# ADHESIVE VASCULAR ACCESS DEVICE DRESSING

## COVADERM PLUS® VAD

### FEATURES & BENEFITS

- Patented\* multi-layer barrier dressing
- Addresses problems associated with covering vascular access sites
- Special "v-cut" tape secures catheter extension tubing and prevents lifting of dressing edge
- Absorbs excess moisture at the site
- Provides a barrier to external contaminants to help prevent infection
- Allows moisture vapor transmission away from site
- Conforms easily to body contours
- Gentle to delicate skin to prevent blistering reactions
- Primary dressing

### INDICATIONS

- Central venous catheter sites
- Subclavian, jugular, femoral and antecubital
- Peripheral I.V. Sites
- Implanted ports
- Midline catheters




\*U.S. Patent No. 5,344,415

The low moisture vapor transmission rate of traditional film dressings can cause the accumulation of moisture at the vascular access site resulting in increased infections rates and blistering reactions on sensitive skin. The Covaderm Plus VAD dressing's unique design eliminates these concerns making it the ideal dressing for covering and protecting all central and peripheral vascular access sites.

### APPLICATION

1. Peel back edge of folded release paper.
2. Anchor exposed edge of tape onto skin, then carefully peel off rest of release paper.
3. Smooth all tape edges onto skin.
4. Peel back edge of release paper on V-cut strip.
5. Place tape under catheter as shown below.

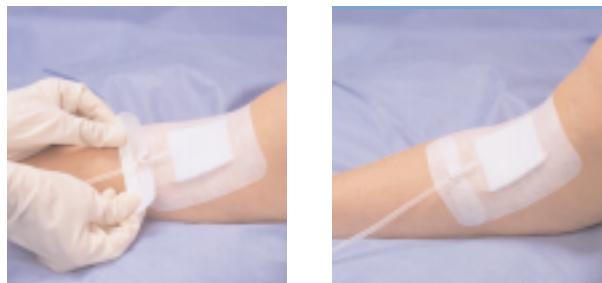

| Product # | Size    | Pad     | V-Tape  | Qty/Bx | Qty/Cs |
|-----------|---------|---------|---------|--------|--------|
| 46-405    | 4" x 4" | 2" x 2" | 1" x 4" | 25     | 200    |
| 46-406    | 6" x 6" | 4" x 4" | 1" x 6" | 5      | 50     |

TO PLACE AN ORDER OR FOR MORE INFORMATION  
 PLEASE CALL: **800-251-9864**  
 865.938.7828 (OUTSIDE USA)  
 800.327.0304 (FAX)



200 DeBusk Lane  
 Powell, Tennessee 37849  
[www.deroyal.com](http://www.deroyal.com)