**medi GmbH & Co. KG**  
 Medicusstraße 1  
 D-95448 Bayreuth  
 Germany  
 T +49 921 912-0  
 F +49 921 912-780  
 ortho@medi.de  
 www.medi.de

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium bvba  
 Posthoornstraat 13/1  
 3582 Koersel  
 Belgium  
 T +32 011 24 25 60  
 F +32 011 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Brasil  
 Rua Neuza 216  
 Diadema-Sao Paulo  
 Cep 09941-640  
 Brazil  
 T +55 11 32011188  
 F +55 11 32011185  
 sac@medibrasil.com  
 www.medibrasil.com

medi Danmark ApS  
 Vejlegardsvej 59  
 2665 Vallensbæk Strand  
 Denmark  
 T +45 46 55 75 69  
 F +45 70 25 56 20  
 kundeservice@sw.dk  
 www.medit danmark.dk

medi Bayreuth España SL  
 C/Canigó 2 – 6 bajos  
 Hospitalet de Llobregat  
 08901 Barcelona  
 Spain  
 T +34 93 260 04 00  
 F +34 93 260 23 14  
 medi@mediespana.com  
 www.mediespana.com

medi France  
 Z.I. Charles de Gaulle  
 25, rue Henri Farman  
 93297 Tremblay en France Cedex  
 France  
 T +33 1 48 61 76 10  
 F +33 1 49 63 33 05  
 infos@medi-france.com  
 www.medi-france.com

medi Hungary Kft.  
 Bokor u. 21.  
 1037 Budapest  
 Hungary  
 T +36 1371 0090  
 F +36 1371 0091  
 info@medi.hu  
 www.medi.hu

medi Medical Support Sdn Bhd  
 medi representative office Asia  
 Unit No. B-2-19, Block B, No.2,  
 Jalan PJU 1A/7A  
 Oasis Ara Damansara, PJU 1A,  
 47301 PETALING JAYA  
 Selangor Darul Ehsan  
 Malaysia  
 T: +6 03 7832 3591  
 F: +6 03 78323921  
 info@medi-asia.com  
 www.medi-asia.com

medi Middle East  
 P. O. Box: 109307  
 Abu Dhabi  
 United Arab Emirates  
 T +971 2 6429201  
 F +971 2 6429070  
 info@mediuae.ae  
 www.mediuae.ae

medi Nederland BV  
 Heusing 5  
 4817 ZB Breda  
 The Netherlands  
 T +31 76 57 22 555  
 F +31 76 57 22 565  
 info@medi.nl  
 www.medi.nl

medi Polska Sp. z o. o.  
 Zygmunt Starego 26  
 44-100 Gliwice  
 Poland  
 T+48 32 230 60 21  
 F+48 32 202 87 56  
 info@medi-polska.pl  
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda  
 Rua Eugénio de Castro Rodrigues, n.º  
 1700-184 Lisboa  
 Portugal  
 T +351 21 843 71 60  
 F +351 21 847 08 33  
 medi.portugal@medibayreuth.pt  
 www.medi.pt

medi RUS LLC  
 Business Center NEO GEO  
 17 Butlerova str.  
 Moscow 117676  
 Russia  
 T +7 (495) 374-04-56  
 F +7 (495) 374-04-56  
 info@medirus.ru  
 www.medirus.ru

medi UK Ltd.  
 Plough Lane  
 Hereford HR4 0EL  
 United Kingdom  
 T +44 1432 37 35 00  
 F +44 1432 37 35 10  
 enquiries@mediuk.co.uk  
 www.mediuk.co.uk

medi Ukraine LLC  
 Tankova Str. 8, office 35  
 Business-center "Flora Park"  
 Kiev 04112  
 Ukraine  
 T +380 44 591 1163  
 F +380 44 455 6181  
 office@medi.ua  
 www.medi.ua

medi USA L.P.  
 6481 Franz Warner Parkway  
 Whitsett, N.C. 27377-3000  
 USA  
 T +1 336 4 49 44 40  
 F +1 888 5 70 45 54  
 orders@mediusa.com  
 www.mediusa.com

**medi**

# Genumedi®

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---

## Intended purpose

Genumedi is a knee support with patella silicone ring. The product is exclusively to be used for the orthotic fitting of the knee and only on unbroken skin.

---

## Areas of use

- Inflammation of the joint (Arthritis)
- Painful joint wear (arthrosis)
- Irritation of the joint (e.g. excessive stress, injuries, operations) linked to swelling and effusion
- Pain in the kneecap (femoropatellar pain syndrome, anterior knee pain)

## Side-Effects / Contraindications

No general side effects are known. However, if supports (or similar aids) are too tight, local pressure effects or constriction of blood vessels or nerves may occur in individuals. If you have any of the following complaints please consult your doctor before using the brace:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).
- Sensitivity and perfusion disorders of the legs or feet (e.g. in the case of diabetes).
- Lymph outflow disorders, as well as undiagnosed swellings of soft tissues, apart from the area of use.

Any combination with other products (e.g. compression stockings) must be discussed in advance with the responsible doctor.

## Mode of Action / Comment on Wear

The integrated silicone ring of the Genumedi reduces stress on the kneecap. Together with the compression of the knitted fabric, it causes a massage effect. This enhances perfusion of the joint and decreases swelling and effusion. Irritation and pain are alleviated. At the same time, muscular control and inherent stabilisation of the knee joint are improved.

Supports are most beneficial during physical activity. The support can in principle be worn throughout the day. This must depend on how comfortable the wearer is. Moreover, the knee may not be bent by more than 70° when sitting. The support should be removed for long breaks (e.g. long periods sitting, long drives or sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and contact your doctor or specialised shop.

The support is suitable for use in water. After use, please rinse thoroughly

## Putting on / Taking off

- Grasp the support with the thumb on the internal upper end of the lateral knob adhesive bands (Fig. 1)
- Pull the support over the knee in such a way that the silicone ring covers the kneecap centrally and free of pressure (Figs. 2 +3).
- To take off the support, grasp the lower lateral rods and pull it downwards and away (Fig. 4).

Please take care that trained staff instruct you when putting on the support for the first time.

---

**Care instructions**

The silicone ring does not need to be removed before washing. (See figs. 5 on how to re-insert). Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

**Note on Storage**

Must be stored dry and at room temperature. Must be protected from the sun and moisture.

**Composition of the Material**

Polyamide, polyester, elasthan

**Statutory Defects Liability/Contractual Guarantee**

Consistent with legal guidelines.

**Disposal**

The product can be disposed of in the domestic waste.

**Important Notes**

This product is made for single patient use only. If the support is used for the treatment of more than one patient or is improperly modified, the product liability of the producer lapses according to the Medical Product Law. If you feel too much pain or an unpleasant feeling when wearing the support, take it off at once and contact your doctor or your specialist provider.

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