E

1- DESCRIPCIÓN

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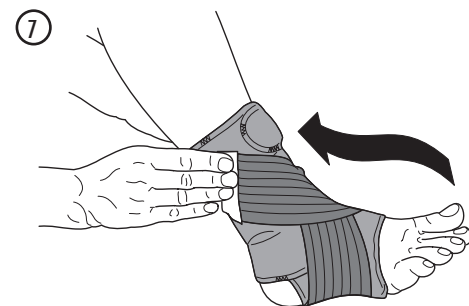
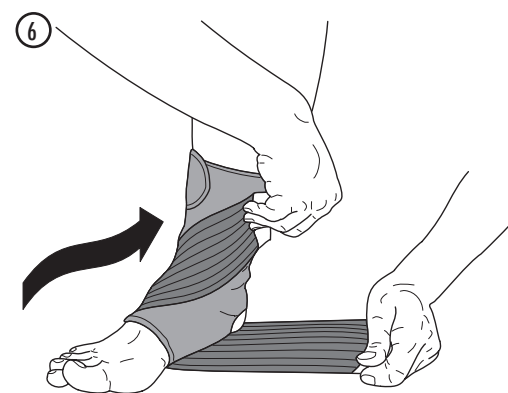
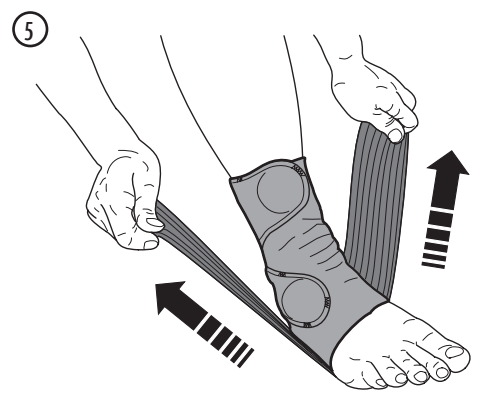
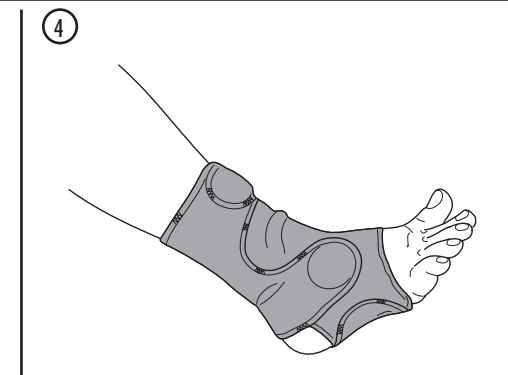
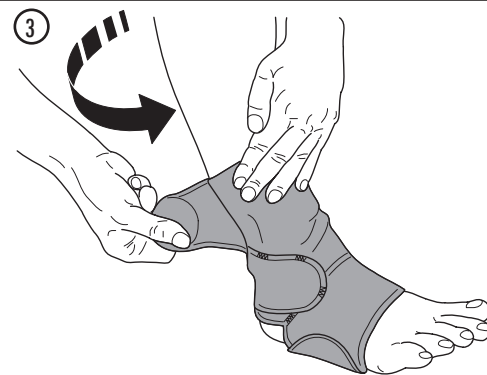
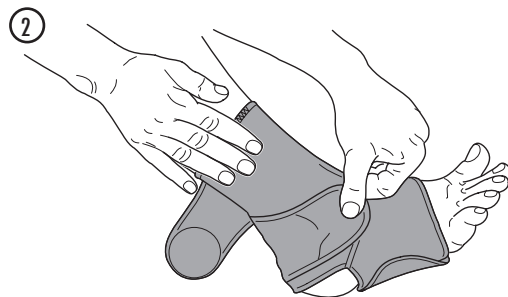
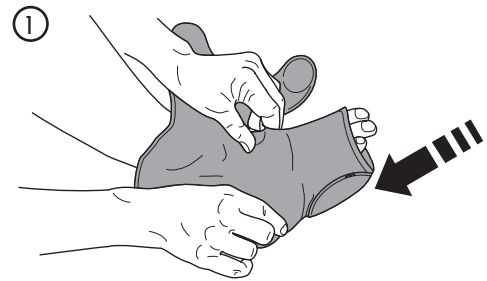
UK

1- DESCRIPTION

This Ankle Lock has been designed and produced as a support and protective device for muscular damages, as well as for tendon and joint injuries either caused by trauma, stress or excessive fatigue.

For the development of this device, two special elastic fabrics have been matched:

- The inside one is made with a very soft cotton velour providing the skin with an extremely comfortable touch and, at the same time, allowing it to breathe;
- The fabric on the outside, made with a particular and dense nylon mesh forming microscopic and resistant loops, allows the velcro pads to securely fasten onto the surface.

Due to this specific construction, a vast range of closing positions are available, depending on the support desired. The strap therefore allows to increase, in a remarkable way, the immobilization and the support of the joint. This Ankle Lock is manufactured in accordance with the primary requirements set under the European Directive 93/42/EEC on medical devices and in accordance with the Italian law (LD 24th February 1997, No. 46) introducing the same EU Directive.

2- INDICATIONS

- Traumatic pathologies: distortions, tendonitis, luxations.
- Rehabilitation phase: Passive support to functional rehab of the ankle.

2-1 INSTRUCTIONS

Wear directly on the skin and follow the diagrams:

3- SIDE EFFECTS AND CONTRA-INDICATIONS

Assuming that the products are used according to instructions and given that they are non invasive, there are no side effects affecting the organism as a whole. All braces and support, if overly tight, can cause discomfort due to compression. If you observe any change in your general state or deterioration of your condition, please consult your doctor. During a prolonged period of inactivity, it is recommended to reduce the tension on your device until you resume normal activities where you will reset the tension to the original level recommended to you.

Any modifications of these products will result in the annulment of the warranty along with the denial of any responsibilities on our behalf.

4- MAINTENANCE AND CARE

- No special maintenance is required.
- Fold the Ankle Lock and attach the velcro pads firmly to the surface.
- Wash gently at lukewarm temperature.
- Do not use any substances containing chlorine.
- Dry at low setting.
- Washing does not damage the product and it actually restores the elasticity.

This product is not biodegradable.

AT THE END OF ITS USABLE LIFE, DISCARD IT ALONG WITH YOUR NORMAL WASTE.

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