1- DESCRIPCIÓN

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#17

Distributed by
MIH International LLC
112 Capitol Trail
Newark, DE 19711 - USA

North America Toll Free: Tel: 866-374-4050
Fax: 866-374-4051
Email: sales@m-brace.com
www.m-brace.com

Made in Italy by
INDACO s.r.l.
Contrada Iscalunga snc
85020 Filiano - Potenza
Italy

UK SALTO ANKLE STABILIZER TO CONTROL INVERSION AND EVERSION

Declaration of conformity (Directive 93/42/EEC, Annex VII)

INDACO srl - Hereby declares that the SALTO ANKLE STABILIZER complies with the provisions transposing the Directive on MEDICAL DEVICES (93/42/EEC).
GENERAL INFORMATION AND INSTRUCTIONSPage 3

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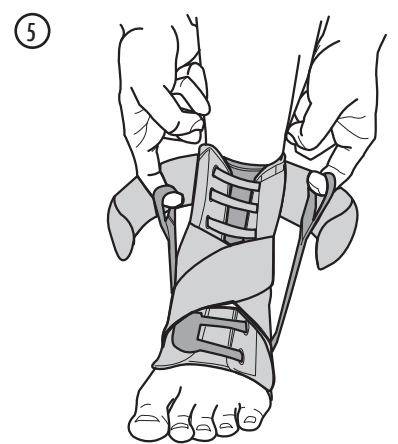
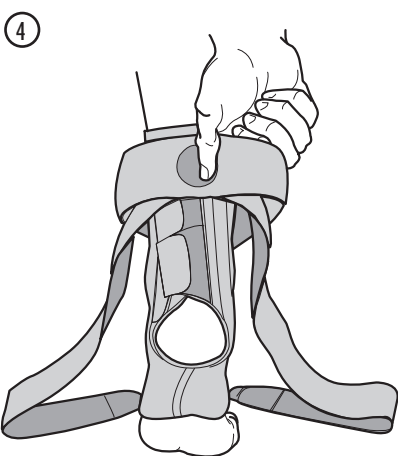
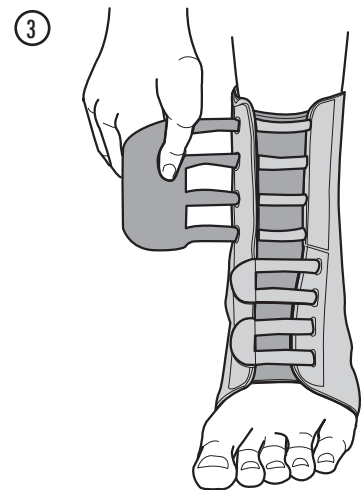
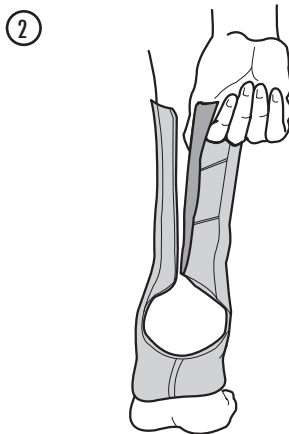
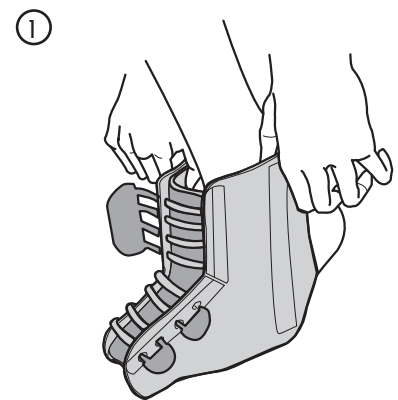
1- DESCRIPTION

This brace was made to stabilize the ankle after a moderate to acute sprain and to guarantee an effective prevention of inversion and eversion movements that could worsen the injury. It is composed of two parts: a low profile support sock with posterior opening for an easy fit and an adjustable locking system that emulates a figure 8 taping technique to control the wanted degree of inversion and eversion. To ensure maximum comfort especially during extended use, this product was made with a breathable fabric. The SALTO Ankle Stabilizer was made in accordance with the primary requirements set under the European Directive 93/42/EEC and with the Italian law (LD 24th February 1997, No. 46) introducing the same EU Directive.

2- INDICATIONS

Indicated for inversion and eversion control of the ankle, instability and sprains of the ligaments, during rehab, post-op or post-trauma.

2-1 INSTRUCTIONS

Apply directly on the skin following the directions on the drawings.

3- SIDE EFFECTS AND CONTRA-INDICATIONS

Assuming that the products are used according to instructions and given that they are non invasive, they are no side effects affecting the organism as a whole. All braces and support, if overly tight, can cause discomfort due to compression. If you observe any change in your general state or deterioration of your condition, please consult your doctor. During a prolonged period of inactivity, it is recommended to reduce the tension on your device until you resume normal activities where you will reset the tension to the original level recommended to you. Any modifications of these products will result in the annulment of the warranty along with the denial of any responsibilities on our behalf.

4- MAINTENANCE AND CARE

Before washing, replace Velcro's in their original position to prevent them from damaging other objects. Wash gently at lukewarm temperature. Do not bleach.

This product is not biodegradable.
AT THE END OF ITS USABLE LIFE, DISCARD IT ALONG WITH YOUR NORMAL WASTE.

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