E

1- DESCRIPCIÓN

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





RIGHT
DROIT
DERECHA
DIREITA

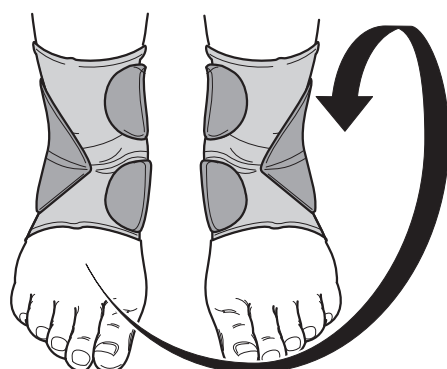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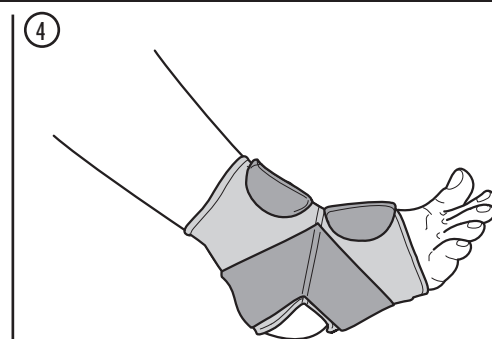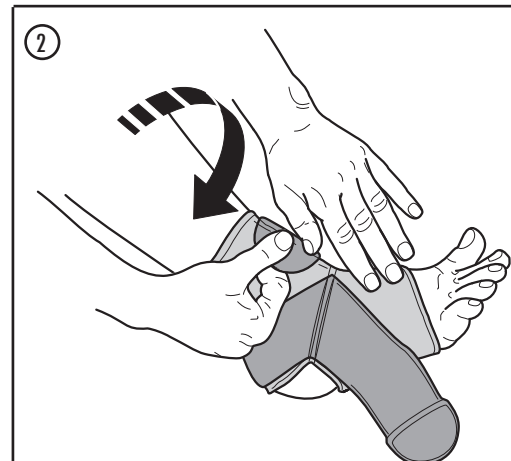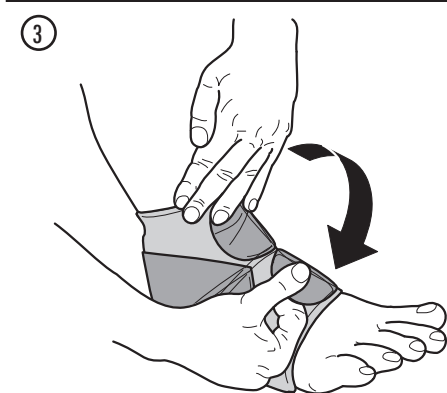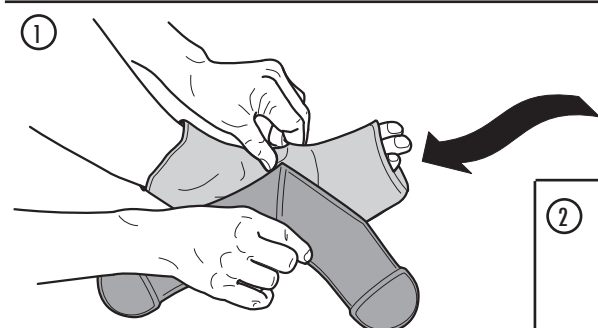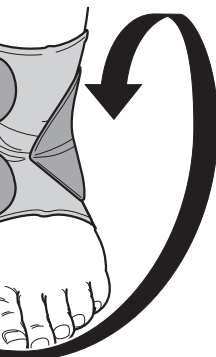1- DESCRIPTION

This Extra Ankle Lock has been designed and produced as a support and protective device for muscular and ligament damages, as well as for tendon and joint injuries either caused by trauma, stress or excessive fatigue. For the development of this device, a special elastic fabric has been used. This fabric is made with a particular and dense nylon mesh forming microscopic and resistant loops, allowing the Velcro pads to securely fasten onto the surface. Due to this specific construction, based on two very strong and supportive straps, a vast range of locking positions are available, depending on the support desired. The double reinforced "high-tension" elastic straps system allows to increase, in a remarkable way, the immobilization and the support of the joint and helps preventing and/or healing the ankle inversion. This Extra Ankle Lock is manufactured in accordance with the primary requirements set under the European Directive 93/42/EEC on medical devices and in accordance with the Italian law (LD 24th February 1997, No. 46) introducing the same EU Directive.

2- INDICATIONS

- Traumatic pathologies: Distortions, tendonitis and sprains.
- Rehab phase: passive support for functional rehabilitation of the ankle.
- Good support during sport.

2-1 INSTRUCTIONS

Wear in direct contact with the skin (or over a sock) and follow the diagrams:

3- SIDE EFFECTS AND CONTRA-INDICATIONS

Assuming that the products are used according to instructions and given that they are non invasive, they are no side effects affecting the organism as a whole. All braces and support, if overly tight, can cause discomfort due to compression. If you observe any change in your general state or deterioration of your condition, please consult your doctor. During a prolonged period of inactivity, it is recommended to reduce the tension on your device until you resume normal activities where you will reset the tension to the original level recommended to you.

Any modifications of these products will result in the annulment of the warranty along with the denial of any responsibilities on our behalf.

4- MAINTENANCE AND CARE

- No special maintenance is required.
- Fold the Extra Ankle Lock and firmly attach the Velcro pads to the surface.
- Wash gently at lukewarm temperature.
- Dry at low setting.
- Washing does not damage the product and it actually restores its elasticity.
- Do not use any substances containing chlorine.

This product is not biodegradable.
AT THE END OF ITS USABLE LIFE, DISCARD IT ALONG WITH YOUR NORMAL WASTE.

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