

NECLOC® EXTRICATION COLLAR

SIZING & APPLICATION INSTRUCTIONS

To be used only as directed by local Emergency Medical Service Authorities.

1. Select proper size.

ADULT SIZES

NL-200EL
Stout

NAVY



NL-250E
XS

GREEN



NL-500E
Large

LT. BLUE



NL-400E
Medium

ORANGE



NL-300E
Small

YELLOW



PEDIATRIC SIZES



NL-P1

0 - 2 yrs



NL-P2

2 - 6 yrs



NL-P3

6 - 12 yrs


grey
pink
red
purple
yellow

white
blue
orange


green

QUICK SIZE CHECK:

Use fingers to measure vertical chin-to-shoulder distance.

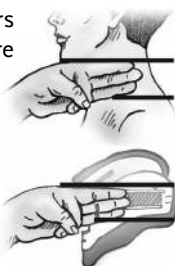


2. Assemble Front/Back Piece (where needed).

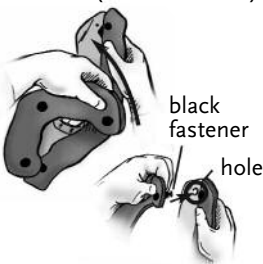

3. Use Cervical Spine Precautions.

INDIVIDUAL'S
NEUTRAL
ALIGNMENT:

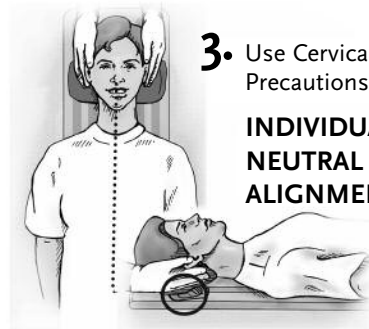

4. Apply Front. SCOOP IT UP!


Attach Retaining Strap.



5. Apply Back symmetrically.



6. Tighten straps.

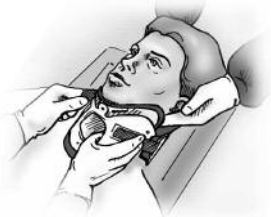

For children 12 years and under:
Maintain appropriate height padding under thorax to offset occipital prominence.



7. Check fit.



Too Small



Too Large



Just Right!

8. Cleaning.

Use a 1:10 dilution disinfectant : water



AUTHORIZED REPRESENTATIVE
Ossur Head Office
Grjothals 5, 110 Reykjavik
Iceland

Ossur Americas
Trauma & Spine
1414 Metropolitan Avenue
Paulsboro, New Jersey 08066

TOLL-FREE 800 257 8440
WEB www.ossur.com



Life Without Limitations