


# NINJA Spinal System


## APPLICATION INSTRUCTIONS

CP020513 Rev A 07/10 L


This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient. This device is not intended to prevent injury or reduce or eliminate the risk of injury.

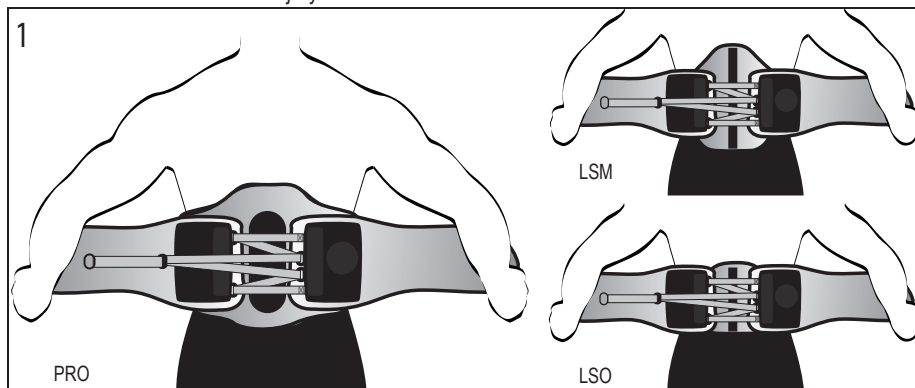

1. Open and extend the corset. The corset should be fully extended each time it is applied. The panel with the handle should be on the left. Center the two ends of the device on the lower back.

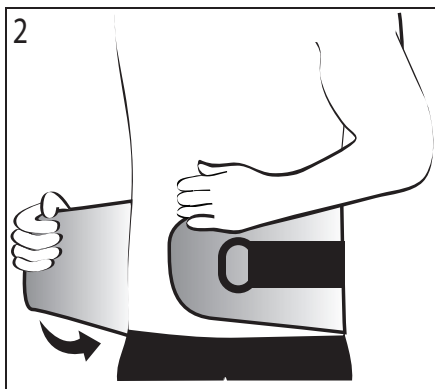

2. Wrap the sides around the abdomen with the right side overlapping the left.

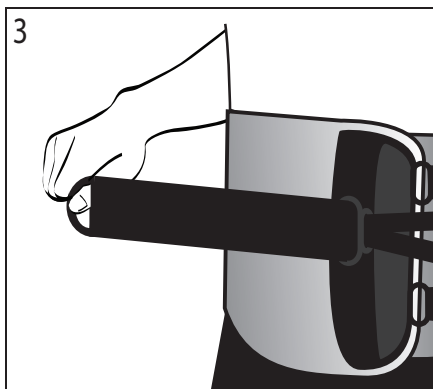

3. Pull the handle to the desired position. When the appropriate amount of tension is reached, place the handle on the Velcro strap to secure.

**INDICATIONS:** Indicated for acute and chronic back pain, compression fractures, degenerative disc disease, herniated or bulging disc, kyphonic or lordotic posture, ligament strains or sprains, osteoporosis, post-operative support, post-laminectomy, spinal metastasis, spinal stabilization, spinal stenosis, spondylolithesis, spindylolysis.

**WARNINGS:** Do not leave this device in a car on a hot day or in any other place where the temperature may exceed 150°F (65°C) for any extended period of time. If any additional pain or symptoms occur while using this brace, discontinue use and seek medical attention.

**WARRANTY:** This device is warranted to be free from defects in material and workmanship for a period of 90 days; and for the life of the brace on the mechanical advantage system. This warranty applies to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at [www.BledsoeBrace.com](http://www.BledsoeBrace.com), or by calling 1.888.BLEDSOE (253.3763).



Council Directive 93/42/EEC  
of 14 June 1993 concerning Medical devices

MANUFACTURED BY:  
© 2010 Medical Technology, Inc. All Rights Reserved d/b/a  
**BLEDSOE BRACE SYSTEMS™**  
2601 Pinewood, Grand Prairie, Texas, 75051, USA  
Toll Free Tel: 1.888.BLEDSOE (253.3763) • 1.800.527.3666  
Local: 972.647.0884 • Local Fax: 972.660.5495  
[www.BledsoeBrace.com](http://www.BledsoeBrace.com)

EUROPEAN AUTHORIZED REPRESENTATIVE

|    |     |  |
|----|-----|--|
| EC | REP | EMERGO EUROPE<br>Molenstraat 15<br>2513 BH, The Hague<br>The Netherlands<br>Tel: (+31) 70 345 8570<br>Fax: (+31) 70 346 7299 |
|----|-----|--|