

Ossur North America

27412 Aliso Viejo Parkway
Aliso Viejo, CA 92656, USA
Tel: +1 800 233 6263
Tel: +1 949 362 3883
Fax: +1 949 362 3888
ossurusa@ossur.com

Ossur - Braces and Support

742 Pancho Road
Camarillo, CA 93012, USA
Tel +1 805 484 2600
Fax +1 800 284 4356

Ossur - Custom Braces

19762 Pauling
Foothill Ranch, CA 92610, USA
Tel +1 949 859 4407
Fax +1 800 453 4567

Ossur Spinal

Mid Atlantic Corporate Center
1414 Metropolitan Ave.
P.O. Box 566
Thorofare, New Jersey 08086, USA
Tel: +1 856 848 0525
Fax: +1 856 845 1148
spine@ossur.com

Ossur Canada

12111 Jacobson Way
Richmond, BC
V6W 1L5, Canada
Tel: +1 604 241 8152
Fax: +1 604 241 8153

Ossur Europe

P.O. Box 120
5690 AC Son en Breugel
The Netherlands
Tel: 00800 3539 3668
Tel: +31 499 462840
Fax: +31 499 462841
saleseurope@ossur.com

Ossur Europe

Kundenservice Deutschland
Römerfeldstraße 2
50259 Pulheim, Germany
Tel: +49 (0) 2238 30 58 50
Fax: +49 (0) 2238 30 58 01
info-deutschland@ossur.com

Ossur Nordic

Salagatan 16a
P.O. Box 67
751 03 Uppsala, Sweden
Tel: +46 1818 2200
Fax: +46 1818 2218
info@ossur.com

Ossur International Markets

Grjothals 5
110 Reykjavík, Iceland
Tel: +354 515 1300
Fax: +354 515 1370
international@ossur.com

Ossur Asia Pacific

2 Redbank Road
Northmead NSW 2152
Australia
Tel +61-2-9630-9206
Fax +61-2-9630-9268
info-asiapacific@ossur.com



Ossur Head Office

Grjothals 5
110 Reykjavík, Iceland
Tel + 354 515 1300
Fax +354 515 1366
mail@ossur.com



**CONTACT CLOSURE
ARM SLING**
Premium & Bergshultz Universal



Life Without Limitations

Premium Contact Closure Arm Sling

Part #	Size	U/M
204112	XSmall	1 ea
204113	Small	1 ea
204115	Medium	1 ea
204117	Large	1 ea
204118	XLarge	1 ea

*Add “P” before item number for shoulder comfort pad.

Bergschultz Universal Arm Sling

Part #	Description	U/M
204070	Universal	1 ea



1



3



5



2



4

Contact Closure Arm Sling Premium & Bergschultz Universal

USES

Post surgery, shoulder, wrist, forearm, and elbow injuries, humeral fractures, and to support arm casts.

CAUTION

Patients should be instructed to inspect their shoulder, elbow, hand and this product before and after each use.

Modifications to this product are the sole responsibility of the prescribing physician.

PHOTO CAPTIONS

- A. Shoulder strap
- B. Sling
- C. Contact closures
- D. D-ring

FITTING INSTRUCTIONS

1. Open the sling.
2. Place injured arm into the sling. Place elbow deep into the pocket of the sling at a 90° angle when possible. Ensure that the distal end supports the hand and fingers.
3. Place strap behind the neck and over the opposite shoulder of the injured arm. Position strap on shoulder for comfort.
4. Undo contact closure on the end of strap. Loop through D-ring and secure.
5. Tighten the strap that goes through the buckle to give proper support. Adjustments may be necessary as arm swelling patterns change.

WASHING INSTRUCTIONS

1. The sling can be hand washed using mild detergent and cold water. Do not wash in washing machine. Do not use bleach.
2. Lay sling out flat on a towel and air dry. Do not place in dryer. Avoid direct heat and sun.

