

I N S T R U C T I O N S F O R U S E

Ossur North America
27412 Aliso Viejo Parkway
Aliso Viejo, CA 92656, USA
Tel: +1 800 233 6263
Tel: +1 949 362 3883
Fax: +1 949 362 3888
ossurusa@ossur.com

Ossur - Braces and Support
742 Pancho Road
Camarillo, CA 93012, USA
Tel +1 805 484 2600
Fax +1 800 284 4356

Ossur - Custom Braces
19762 Pauling
Foothill Ranch, CA 92610, USA
Tel +1 949 859 4407
Fax +1 800 453 4567

Ossur Spinal
Mid Atlantic Corporate Center
1414 Metropolitan Ave.
P.O. Box 566
Thorofare, New Jersey 08086, USA
Tel: +1 856 848 0525
Fax: +1 856 845 1148
spine@ossur.com

Ossur Canada
12111 Jacobson Way
Richmond, BC
V6W 1L5, Canada
Tel: +1 604 241 8152
Fax: +1 604 241 8153

Ossur Europe
P.O. Box 120
5690 AC Son en Breugel
The Netherlands
Tel: 00800 3539 3668
Tel: +31 499 462840
Fax: +31 499 462841
saleseurope@ossur.com

Ossur Europe
Kundenservice Deutschland
Römerfeldstraße 2
50259 Pulheim, Germany
Tel: +49 (0) 2238 30 58 50
Fax: +49 (0) 2238 30 58 01
info-deutschland@ossur.com

Ossur Nordic
Salagatan 16a
P.O. Box 67
751 03 Uppsala, Sweden
Tel: +46 1818 2200
Fax: +46 1818 2218
info@ossur.com

Ossur International Markets
Grjóthals 5
110 Reykjavík, Iceland
Tel: +354 515 1300
Fax: +354 515 1370
international@ossur.com

Ossur Asia Pacific
2 Redbank Road
Northmead NSW 2152
Australia
Tel +61-2-9630-9206
Fax +61-2-9630-9268
info-asiapacific@ossur.com



Ossur Head Office
Grjóthals 5
110 Reykjavík, Iceland
Tel + 354 515 1300
Fax +354 515 1366
mail@ossur.com

EQUALIZER®
LOW TOP WALKERS
Premium and Pre-Inflated



Life Without Limitations

Equalizer® Premium Low Top Walker*

Grey Part #	Size	Men Shoe	Women Shoe	U/M
Wo300	XSmall	2 - 4	3.5 - 5.5	1 ea
Wo500	Small	4.5 - 7	6 - 8	1 ea
Wo700	Medium	7.5 - 10.5	8.5 - 11.5	1 ea
Wo900	Large	10.5 - 12.5	11.5 - 13.5	1 ea
W1100	XLarge	12.5+	13.5+	1 ea

*Standard color is Grey. Also available in Black. Add BLK for Black

Premium Low Top Walker Replacement Softgoods

Grey Part #	Black Part #	Size	U/M
WKSG0300	WKSG0300BLK	XSmall	1 ea
WKSG0500	WKSG0500BLK	Small	1 ea
WKSG0700	WKSG0700BLK	Medium	1 ea
WKSG0900	WKSG0900BLK	Large	1 ea
WKSG1100	WKSG1100BLK	XLarge	1 ea

Equalizer® Pre-Inflated Low Top Air Walker (Available in black only)

Black Part #	Size	Men Shoe	Women Shoe	U/M
PW0300	XSmall	2 - 4	3.5 - 5.5	1 ea
PW0500	Small	4.5 - 7	6 - 8	1 ea
PW0700	Medium	7.5 - 10.5	8.5 - 11.5	1 ea
PW0900	Large	10.5 - 12.5	11.5 - 13.5	1 ea
PW1100	XLarge	12.5+	13.5+	1 ea

Pre-Inflated Low Top Air Walker Replacement Softgoods (Available in black only)

Part #	Size	U/M
12923	XSmall	1 ea
12924	Small	1 ea
12927	Medium	1 ea
12928	Large	1 ea
12929	XLarge	1 ea

Hot/Cold Therapy Insert - Gel Pad

Part #	Size	U/M
WKGEL100	Universal	1 ea



1



5



7



2



6



8



3



4



A.
B.
C.
D.
E.
F.

EQUALIZER® LOW TOP WALKER

Premium & Pre-Inflated

USES

Grade 2 and 3 Ankle Sprains, Stable Fractures, Post-Operative Use, Trauma, Rehabilitation, and Hot/Cold Therapy

CAUTION

DO NOT REMOVE WALKER UNLESS INSTRUCTED BY PHYSICIAN.

1. This product has been designed and tested based on single patient usage and is not recommended for multiple patient usage.
2. If skin irritations of any kind develop, consult medical professional immediately.
3. Be sure the patient performs circulation checks. If a loss of circulation is felt, or if the walker feels too tight, release contact closure straps and adjust to a comfortable level. If discomfort continues, the patient should contact their medical professional immediately.
4. Caution should be used if prescribed for patients with diabetes, vascular deficiencies, and neuropathies.
5. Do not remove the inner foam pads which are adhered to the plastic walker boot.

PHOTO CAPTIONS

- A. Pull tab
- B. Adjustable contact closure straps
- C. Walker struts
- D. Softgood
- E. Wide rocker bottom base
- F. Foam pads

FITTING INSTRUCTIONS

1. Remove softgood by opening contact closure straps and lifting out of brace. Do not remove pull tabs at this time.
2. Place lower leg and foot into softgood. Close softgood snugly over leg.
3. Insert foot into brace by gently spreading uprights. Do not remove pull tabs at this time.
4. Ensure struts are centered over the ankle bone (malleoli). The lower leg should be kept at a 90° angle with the foot.
5. Remove pull tabs located in between struts and softgood. This allows the softgood to be secured into position.
6. To accommodate for rubbing or spacing, additional foam pads may be placed inside the softgood, between the softgood and plastic, or under straps for additional comfort.
7. Secure contact closure straps, starting at the toes and working upward. For best fit, secure straps in the correct order. Strap adjustments may be necessary as foot, ankle and leg swelling patterns change.
8. When prescribed by physician, remove leg from product by releasing contact closure straps and opening softgood. Pull foot and leg forward and out, leaving softgood inside the brace. If softgood is removed, follow Steps 1 through 7 to reapply.

WASHING INSTRUCTIONS

1. This product can be washed at any time by hand using a mild detergent and cold water. Remove softgood from walker plastic before washing. Do not use bleach or strong detergent. Do not wash in washing machine or place in dryer. Air dry flat.
2. Walker plastic can be washed with a damp cloth and mild soap.

Equalizer® Low Top Premium Walker and Equalizer® Low Top Pre-Inflated Walker are covered by one or more of the following Patent Nos. 5,078,128. Other U.S. and foreign patents pending. ® indicates registration in U.S.A. and selected countries only.