FOOT-UP™

Product Number : 07810-X
27810-X

Life Without Limitations®

.....	3
(EN) Instructions for use.....	5
(DE) Gebrauchsanweisung.....	9
(FR) Guide de fabrication.....	13
(ES) Instrucciones para el uso.....	17
(IT) Istruzioni per l'uso.....	21
(NO) Bruksanvisning.....	25
(DA) Brugsanvisning.....	29
(SV) Bruksanvisning.....	33
(FI) Käyttöohjeet.....	37
(NL) Gebruiksaanwijzing.....	41
(PT) Instruções de Utilização.....	45
(JA) 取扱説明書.....	49

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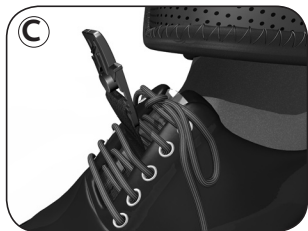
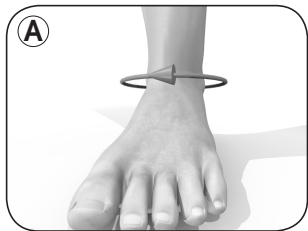
ENGLISH

DEAR CUSTOMER,

We would like to ask you to follow these Instructions for Use with care. If you should have any queries, ask the doctor treating you, at your nearest orthotist or contact us directly.

FUNCTION

The Foot-Up is a lightweight, dynamic aid for drop foot or related disorders that require dorsal flexion support. The Foot-Up basic version consists of two separate parts: an ergonomic ankle strap and a barely visible inlay. The easy-to-fit ankle strap is made from a hypo-allergenic material with outward vaporisation. This reduces the chance of skin reaction. The flexible inlay is placed between the tongue of the shoe and the laces. The position of the inlay determines the degree of support. The two separate components can be connected to each other with the strong easylock elastic strap. This structural combination enables the forefoot to be raised if you need dorsal flexion support, especially in the swinging leg phase, thus approximating to a physiological gait pattern.

ACCESSORIES (Sold separately)

Two accessories are available for the basic version of the Foot-Up:

- 1) A universal shoeless bandage. The bandage makes the Foot-Up suitable for use when not wearing shoes.
- 2) A separate inlay. This makes it unnecessary to change inlays when using different lace-up shoes.

INDICATIONS

- Foot-raising paresis, particularly suitable for KG 3,4.

CONTRA-INDICATIONS

- Circulatory disorders.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage). Generally

speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. welling,reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.

SIDE-EFFECTS

- With proper use and proper fitting, so far there have been no reports of serious general side-effects.
- Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

DETERMINING SIZE: BASIC VERSION

The correct size is determined as follows:

- 1) Measure the circumference of the ankle around the malleoli (**figure A**).
- 2) Find the corresponding size in the table below.

Size	Circumference in cm
Medium	18-21 cm
Large	22-26 cm
X-Large	27-33 cm

FITTING THE BASIC VERSION

The Foot-Up basic version can only be positioned and used in lace-up shoes.

- Detach the inlay from the ankle strap by pressing the button in the middle of the quick-acting closure.
- Apply the ankle strap with the narrow Velcro strip at the botttom and the quickacting closure at the front. In doing this place the ankle strap above the widest part of the ankle. Fasten the ankle strap tightly, but do not constrict the ankle (**figure B**).
- Place the inlay between the tongue and the shoelaces. Then pull the elastic strap between the two lace lines (**figure C**). By pulling the inlay further in the direction of the front of the foot more support is provided when raising the

foot. Make sure the elastic strap is also pulled between the lace lines further down the foot.

- Connect the ankle strap and the inlay by pushing the two parts of the quick-acting closure together (**figure D**).

DETERMINING SIZE: SHOELESS VERSION

The correct size is determined as follows:

- 1) Measure the circumference of the foot immediately behind the ball of the foot (**figure E**).
- 2) Find the corresponding size in the table below.

Size	Circumference in cm
Small	< 20 cm
Medium	20-24 cm
Large	24-28 cm
X-Large	> 28 cm

FITTING THE SHOELESS VERSION

- Apply the ankle strap with the narrow Velcro strip at the bottom and the quickacting closure at the front. In doing this place the ankle strap above the widest part of the ankle. Fasten the ankle strap tightly, but do not constrict the ankle (**figure F**).
- Place the shoeless bandage immediately behind the ball of the foot, so that the button rests on top of the foot (**figure G**).
- Connect the ankle strap and the inlay by pushing the two parts of the quick-acting closure together (**figure H**).

FOOT COMFORT

The anatomical design and the materials used make the Foot-Up both comfortable to wear and userfriendly. The inlay has been designed in such a way that the functionally intended raising of the forefoot can be established without any effort. The Foot-Up is practically invisible, lightweight, and does not have to be adapted to a shoe.

IMPORTANT

- The product is intended for the patient to use just once.
- Improper modifications to the product and /or improper use of the product referred to above exempt the manufacturer from product liability.
- You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.
- For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

CARE

The Foot-Up should be washed in lukewarm water and air-dried. Do not use a tumble dryer.

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