

I N S T R U C T I O N S F O R U S E

Ossur North America
 27412 Aliso Viejo Parkway
 Aliso Viejo, CA 92656, USA
 Tel: +1 800 233 6263
 Tel: +1 949 362 3883
 Fax: +1 949 362 3888
 ossurusa@ossur.com

Ossur - Braces and Support
 742 Pancho Road
 Camarillo, CA 93012, USA
 Tel +1 805 484 2600
 Fax +1 800 284 4356

Ossur - Custom Braces
 19762 Pauling
 Foothill Ranch, CA 92610, USA
 Tel +1 949 859 4407
 Fax +1 800 453 4567

Ossur Spinal
 Mid Atlantic Corporate Center
 1414 Metropolitan Ave.
 P.O. Box 566
 Thorofare, New Jersey 08086, USA
 Tel: +1 856 848 0525
 Fax: +1 856 845 1148
 spine@ossur.com

Ossur Canada
 12111 Jacobson Way
 Richmond, BC
 V6W 1L5, Canada
 Tel: +1 604 241 8152
 Fax: +1 604 241 8153

Ossur Europe
 P.O. Box 120
 5690 AC Son en Breugel
 The Netherlands
 Tel: 00800 3539 3668
 Tel: +31 499 462840
 Fax: +31 499 462841
 saleseurope@ossur.com

Ossur Europe
 Kundenservice Deutschland
 Römerfeldstraße 2
 50259 Pulheim, Germany
 Tel: +49 (0) 2238 30 58 50
 Fax: +49 (0) 2238 30 58 01
 info-deutschland@ossur.com

Ossur Nordic
 Salagatan 16a
 P.O. Box 67
 751 03 Uppsala, Sweden
 Tel: +46 1818 2200
 Fax: +46 1818 2218
 info@ossur.com

Ossur International Markets
 Grjóthals 5
 110 Reykjavík, Iceland
 Tel: +354 515 1300
 Fax: +354 515 1370
 international@ossur.com


Ossur Asia Pacific
 2 Redbank Road
 Northmead NSW 2152
 Australia
 Tel +61-2-9630-9206
 Fax +61-2-9630-9268
 info-asiapacific@ossur.com



GameDay™ Ankle Brace

Part #	Size	Ankle Circ.*	U/M
W-10601	XXSmall	9" - 10"	1 ea
W-10602	XSmall	10" - 11"	1 ea
W-10603	Small	11" - 12"	1 ea
W-10605	Medium	12" - 13"	1 ea
W-10607	Large	13" - 14"	1 ea
W-10608	XLarge	14" - 15"	1 ea
W-10609	XXLarge	15" - 16"	1 ea

* Measure ankle circumference diagonally over the base of the heel and across the ankle bone as shown.

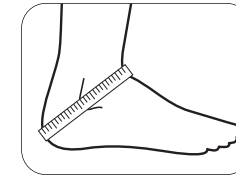

Ossur Head Office
 Grjóthals 5
 110 Reykjavík, Iceland
 Tel + 354 515 1300
 Fax +354 515 1366
 mail@ossur.com



**GAMEDAY™
 ANKLE BRACE**



Life Without Limitations

GameDay™ Ankle Brace

USES

The GameDay Ankle Brace may be used to support mild to moderate ankle sprains and strains, prevent ankle injury or to stabilize chronically weak ankles.

CAUTION

1. Patients should be instructed to inspect their foot, ankle, and this product before and after each use.
2. This product is designed to be worn over an athletic sock. If any problems occur with this product, please contact your certified medical professional.

PHOTO CAPTIONS

- A. Tongue
- B. Laces
- C. Heel opening
- D. Contact closure
- E. Figure-8 straps
- F. Elastic circumferential strap
- G. Finger pull tabs

FITTING INSTRUCTIONS

1. Loosen laces and unfasten all straps.
2. Insert foot into the brace so that the tongue is on top of the foot and the heel is located in the heel opening. **NOTE:** The GameDay is designed to be worn over an athletic sock.
3. With foot slightly flexed, fasten the laces to a snug fit. **SIZING NOTE:** If the gap between the laced edges is more than 2" the product is too small. Use the next larger size (see sizing chart on previous page).

4. Wrap the inside (medial) figure-8 strap across the top of the foot, under the heel, and fasten using the contact closure. Repeat for the outside (lateral) figure-8 strap. Adjust straps for a comfortable and supportive fit.
5. Wrap the elastic circumferential strap around ankle, covering laces and ends of figure-8 straps.
6. During athletic activity the GameDay can be quickly adjusted to optimize comfort and support: Unfasten the elastic circumferential strap and use the finger pull tabs to adjust the tension on the figure-8 straps. Reapply the elastic circumferential strap and continue activity.

WASHING INSTRUCTIONS

Machine wash gentle cycle in cold water and mild soap with straps and laces fastened. Do not use bleach or strong detergents. Air dry only. Do not place in a dryer or near a heater.



1



4



2




5



3



6

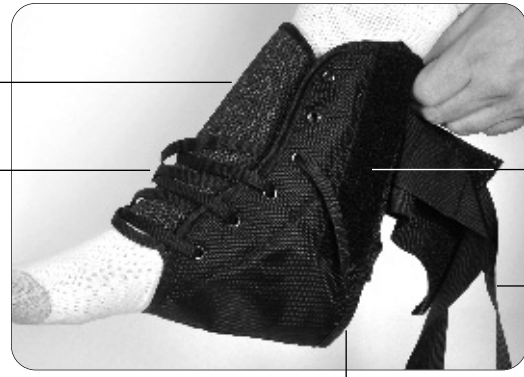

A.

B.

C.

D.

E.

G.

F.