Instructions for use

Innovator Sized Model

Life Without Limitations®

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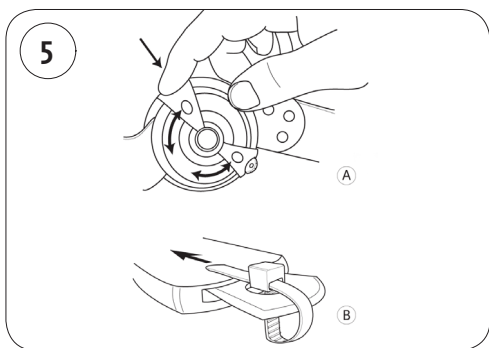
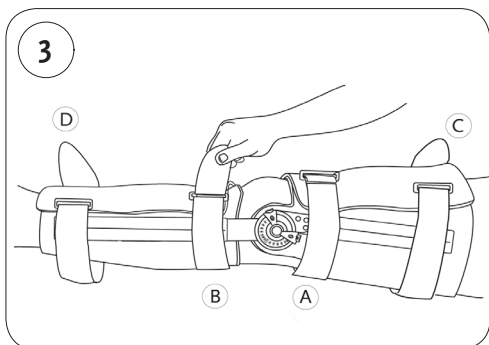
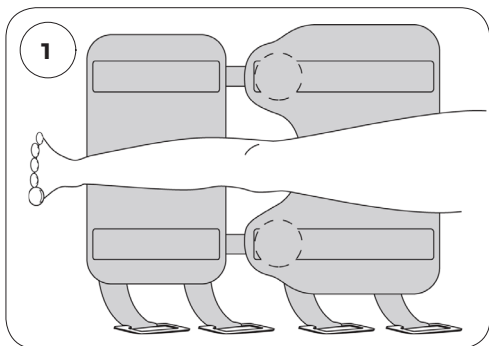
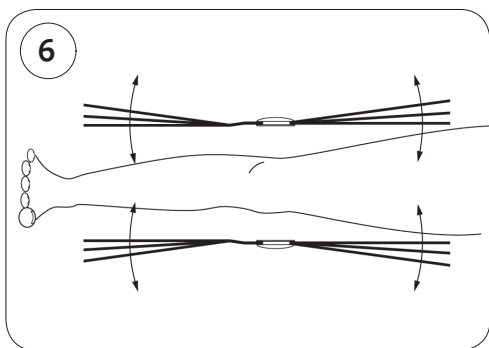
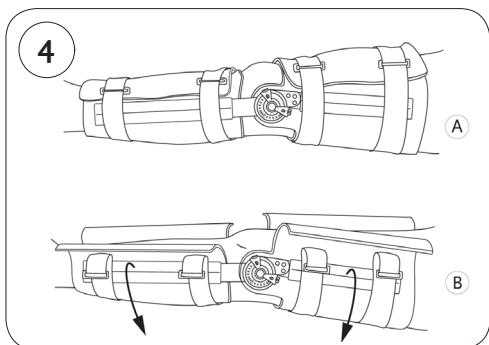
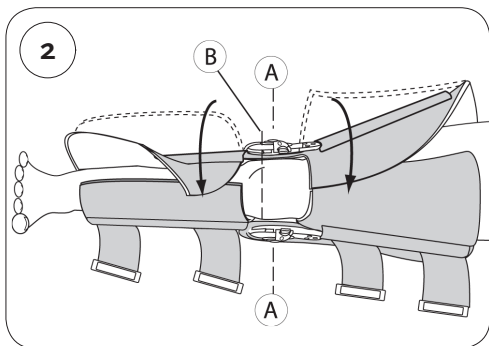
ENGLISH

APPLY THE FOAM WRAPS

1. Open the foam cuffs and straps and slide the brace under the afflicted leg (Figure 1).

ALIGN AND APPLY HINGES

1. Locate hinge pivot center (Figure 2 a) slightly above the joint midline (Figure 2 b).
2. Wrap foam pads around upper and lower leg respectively.
3. Ensure the patella remains exposed.

SECURE THE LEG STRAPS

1. Secure and fasten all straps, starting with the straps closest to the knee first (Figure 3a, b, c, d).

BRACE APPLICATION AND REMOVAL

A properly applied brace (Figure 4a) will have:

1. Foam wraps secured at the front of the leg.
2. Hinges will be aligned vertically with each other at the midline of the leg, and pivot just above the knee axis.
3. All straps will be secured at the front of the leg.

Removal and Re-application:

4. To remove the brace, unfasten the straps and open the foam pads (Figure 4b). The brace then can be removed and reapplies as a single unit.

HINGE OPERATION AND ADJUSTMENT

1. To adjust the hinge, press the flexion or extension button and rotate until the desired degree of flexion or extension shows through the corresponding window and release button (Figure 5a). **The “O-lock” settings are used to lock the brace in full extension.**

Enforced Compliance:

2. Loop the enclosed lock ties through the hole in the flexion and extension buttons limit range of motion (Figure 5 b).

SIDE BAR ADJUSTMENT

1. Side bars can be bent to fit individual legs (Figure 6).

USE AND CARE

1. Read ALL instructions completely before use.
2. Hand wash foam liners with mild soap.
3. All foam pads may be trimmed as necessary.
4. Air dry only, do not heat dry.

CAUTIONS AND WARNINGS

1. Carefully read use instructions and warnings prior to operation.
2. Federal law restricts this device to sale by or on the order of a licensed health care practitioner.
3. For single patient use only.
4. If you experience increased pain, swelling, sensation changes or any adverse reactions while using this product, immediately consult your medical professional.
5. This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also essential parts of a safe treatment program.

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® indicates trademark registration in U.S.A. and selected countries only.