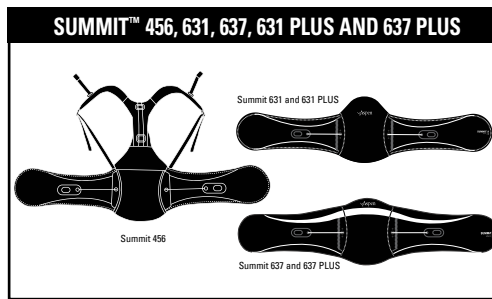
# SUMMIT LINE

Aspen Medical Products

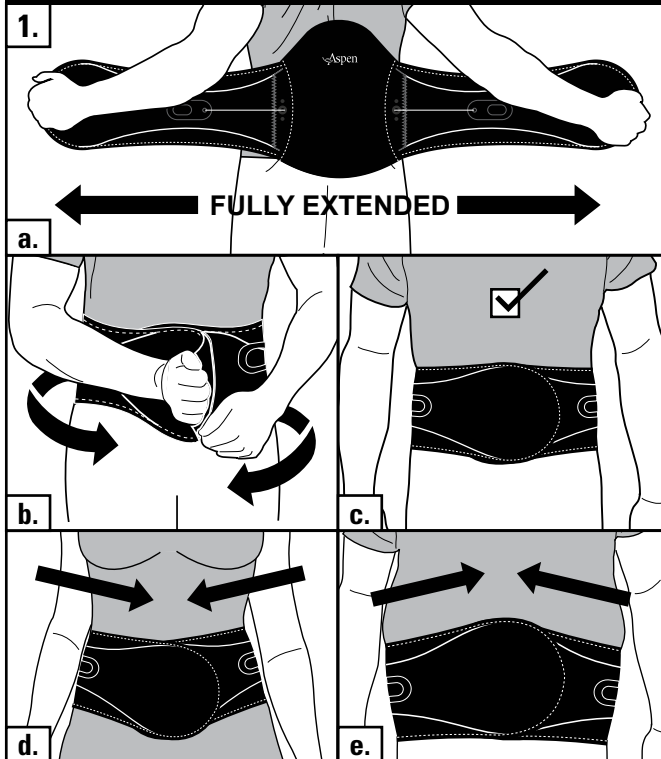
6481 Oak Canyon, Irvine, CA 92618-5202, USA

800-295-2776 • 949-681-0200

aspenmp.com

## SUMMIT® 631, 637, 631 PLUS AND 637 PLUS

1.

a.

b.

d.

c.

e.

### ENGLISH

#### PLACEMENT

- Place the pull-tabs 2 inches from the plastic in the back of the Summit. Pull the ends of the Summit away from each other ensuring that it is fully extended.
- Center over your lower back and apply the brace tightly around the abdomen, the right side overlapping the left.
- When applied properly, the Summit Logo will be visible. (The rigid anterior panels do not need to align perfectly in front of the brace.)
- For patients with a larger hip to waist ratio, the ends of the Summit should meet at a downward angle to fit properly and limit migration.
- For patients with pendulous abdomens, the ends of the Summit should meet at an upward angle to fit properly and limit migration.

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### ENGLISH

#### TIGHTENING

- Slip your thumbs through the holes in the pull-tabs. Pull away from your body.
- The right pull-tab controls the upper compression and the left pull tab controls lower compression. Adjust the compression as needed. When tightened, secure the pull-tabs to the hook-and-loop in front of the brace.

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**ENGLISH**  
**BACK PANEL**  
Custom form the malleable aluminum support in the back panel to properly fit patient anatomy.

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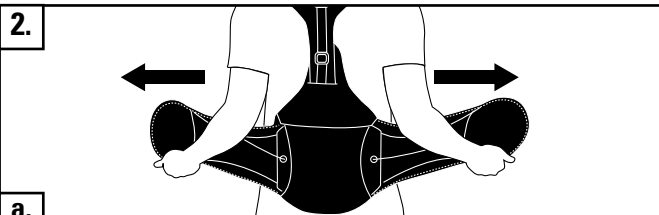
**ENGLISH**  
**PLACEMENT**  
a. Apply the fully stretched brace making sure the back panel is centered and low on the back.  
b. Overlap the ends of the brace so the hook material grips the opposite side loop as tightly as possible.  
c. Ends may be angled to accommodate for different anatomies.

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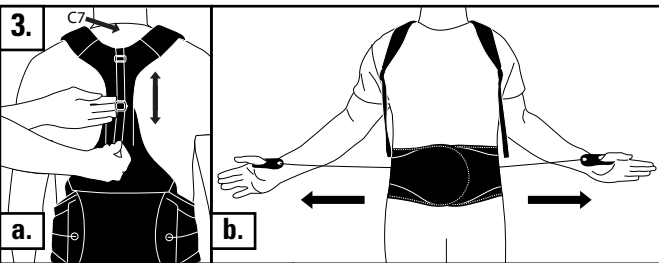
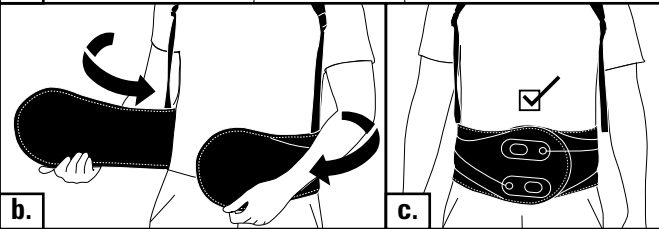
**ENGLISH**  
**TIGHTEN AND ADJUST**  
a. Adjust the vertical height of the posterior adjustment strap so that the yoke rests at C7, or 2" below the shirt collar.  
b. Slip each thumb into the pull tabs. Extend the tabs out and away from the brace. When desired compression is achieved, place the hook side of the pull tabs down onto the loop material to secure.  
c. Pull and tighten the shoulder straps so they fit wide on the shoulders.  
d. Fasten and tighten the chest buckle to secure the shoulder straps.

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<b>SUMMIT</b> <b>LIVE</b>	<b>Small</b>	<b>Medium</b>	<b>Large</b>	<b>Extra Large</b>
	26 to 34 in. (66 to 86 cm)	33 to 42 in. (84 to 107 cm)	41 to 51 in. (104 to 130 cm)	50 to 60 in. (127 to 152 cm)

\* The Aspen Summit™ 456, 631, 637, 631 PLUS and 637 PLUS Brace sizing follows closely to standard pants sizing. Size ranges are only guidelines. Your experiences and personal assessment could make the difference in choosing the correct size.

**LIMITED WARRANTY**  
Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or part(s).  
Aspen Medical Products shall have no obligation under this limited warranty in the event:  
(a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;  
(b) The product is altered;  
(c) Any parts not supplied by Aspen Medical Products are inserted into the product; or  
(d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.  
This warranty gives you specific legal rights and you may have other rights which vary from state to state.  
Warning: To ensure proper use, please review all material.  
A listing of authorized distributors in your area is available upon request.

**READ INSTRUCTIONS BEFORE USE.** Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.