Instructions for use

EN	Instructions for use	1
DE	Einbauanleitung	4
FR	Guide de fabrication	8
ES	Instrucciones para el uso	12
IT	Istruzioni per l'uso	16
DA	Brugsanvisning	20
SV	Bruksanvisning	24
NL	Gebruiksaanwijzing voor de vervaadiging	28
PO	Instruções de Utilização	32
JA	取扱説明書	36

Unloader® One

Life Without Limitations

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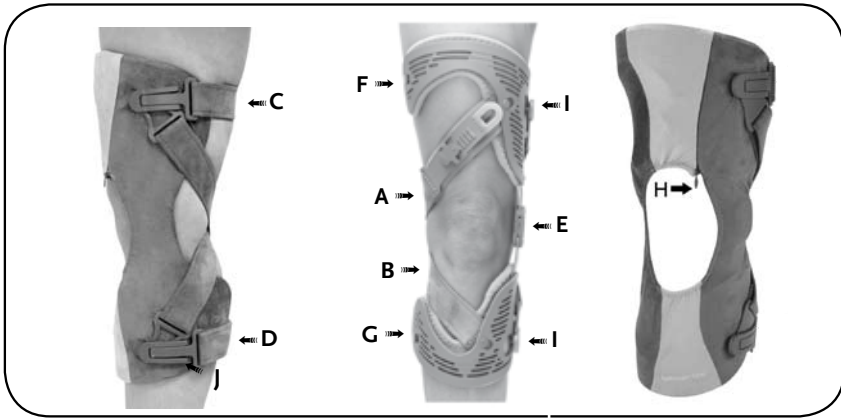
Unloader[®] One Indication for Use:

- Mild to Severe Unicompartmental Osteoarthritis
- Unicompartmental knee conditions that require load reduction to affected compartment such as:
 - Articular cartilage defect repair
 - Meniscal cartilage repair
 - Avascular necrosis
 - Tibia plateau fracture
- Normal activities of Daily living
- Circumference limitation
 - Thigh circumferences smaller than 13.5" (34.3 cm) or bigger than 29" (73.5cm).
 - Calf circumferences smaller than 10" (25cm) or larger than 24" (61.5cm).

Caution: For patients with peripheral vascular disease, neuropathy, and sensitive skin, close physician supervision is recommended.

The Unloader[®] One pain relief device is field adaptable to accommodate patient comfort issues. Modifications to flexion or extension stops and shell contour should only be made by the health care professional. The healthcare professional is responsible for advising patient on use of the Dynamic Force Control System system.

One-time Strap length adjustment:

Thigh and Calf straps

1. Place brace on leg in full extension with center of hinge at center of patella and mid line of anterior / posterior plane.
2. Starting at calf (with Quick Fit Buckle in closed position) bring strap firmly across back of leg.
 - a. Note distance knob on bottom of Quick Fit Buckle extends past keyhole in shell (Picture 1).
 - b. Loosen end of calf strap from Quick Fit Buckle and shorten strap this distance (Picture 2).
 - c. Open Quick Fit Buckle and insert knob into keyhole above buckle-stability pin (click into narrow end) (Picture 3).
 - d. Close Quick Fit Buckle carefully (Picture 4).
 - e. Check tightness - if additional adjustment is needed - continue steps 2a to 2d, until optimum tightness is achieved.
3. Repeat step 2 with same procedure for thigh strap.

Dynamic Force Straps (DFS)

- a. Make sure both thigh and calf straps are adjusted in their locked positions and device correctly positioned on leg in full extension.
- b. With Dynamic Force Control System set at 0 open both Quick Fit Buckles and remove from their keyholes.
- c. Starting with upper DFS adjustment, remove DFS strap from lower Quick Fit Buckle and reinsert the opposite way into D ring of Quick Fit Buckle (Picture 5).
- d. Insert knob of lower Quick Fit Buckle into keyhole and close carefully
- e. With leg in full extension pull on end of DFS strap until snug on leg - hold in this position.
- f. Move Velcro piece and place on strap at D ring feature of Quick Fit Buckle, still holding strap snug on patient's leg (Picture 6).

- g. Re-open Quick Fit Buckle and remove from keyhole
- h. Re-insert DFS strap into D ring of Quick Fit Buckle the correct way and fold at end of hook to secure strap (Picture 7).
- i. Insert button end of lower Quick Fit Buckle into keyhole and close carefully
- j. To adjust lower DFS strap repeat steps 4c & 4d for upper Quick Fit Buckle
- k. In a sitting position with knee at 80° angle and foot flat on floor, pull on end of DFS strap until snug on leg
- l. Repeat steps 4f to 4i for upper Quick Fit Buckle
- m. Fully extend knee and make sure that brace sits well on leg.
- n. Putting knee back to 80° of flexion, open zipper on liner cover and pull on Dynamic Force Control System until arrow points at number 5 (Picture 8).
- o. Fully extend leg and close zipper on sleeve

NOTE: Have patient wear brace for a few minutes to make sure straps are at comfortable tension. Pressure reducing pads are included and may be applied for additional comfort.

Cut Straps:

- a. Remove brace from patient's leg.
- b. Cut excess length from strap leaving enough material for any future adjustments.
- c. Make sure ends of hook Velcro are not overlapping strap - exposed Velcro can be abrasive

Liner Cover applications:

For any required removal of liner cover because of washing or fitting adjustments, replace liner cover with extra set of liners that comes with each brace.

Removing liner cover

1. Remove small knob on end of thigh and calf straps from keyholes (Picture 9).
2. Remove liner cover from device by freeing first thigh section, then calf section.
3. Be careful not to damage liner cover during the process by pulling too hard.

Applying liner cover

1. Apply liner cover to device by starting at calf section and wrapping it around calf shell, placing it correctly by using Velcro attachments on shell.
2. Pull thigh section of liner cover over device and wrap it around thigh shell, attaching with Velcro on shell.
3. Reattach thigh and calf straps to keyholes by threading button hook on strap through hole on inside of liner cover, find small keyhole with your finger and pull button hook into a locked position.
4. **MAKE SURE THAT UPPER DFS STRAP IS UNDERNEATH LOWER DFS STRAP.**

Shell adjustment:

1. Remove liner cover from brace and replace straps back in keyholes.
2. Use heat gun to heat specific area of shell.
3. Apply heat from inside of shell.
4. Replace liners back in brace. Place brace on patient's leg in correct position.
5. Tighten straps.
6. Let cool on patient's leg.
7. Replace liners with liner cover.

Donning Brace (straps have been adjusted to its standard setting)

1. Align brace on patient's leg with center of hinge at mid patella and midline of a / p plane.
2. With both Quick Fit Buckles open - place button of lower Quick Fit Buckle into appropriate keyhole above buckle-stability pin and close carefully.
3. Bend knee to an 80° angle - foot flat on floor.
4. Place button of upper Quick Fit Buckle into appropriate keyhole and close carefully.
5. Open zipper on sleeve.
6. Tighten Dynamic Force Control System to a prescribed position: Practitioner will need to determine optimum unloading based on patient's pain relief feedback but initial setting is recommended at 5.
7. Fully extend leg in a sitting position and make sure that brace is still aligned correctly on leg.

8. If patient requires more or less unloading, adjust the tension of upper DFS strap accordingly with Dynamic Force Control System and/or lower DFS strap by adjusting length at thigh Quick Fit Buckle.
9. If Dynamic Force Control System has been pulled to its maximum and patient requires more pain relief, DFS straps need to be shortened accordingly on lower Quick Fit Buckle.
10. When optimum tightness is achieved, close zipper on liner cover with leg in full extension.

Removing Brace

1. Open zipper on liner cover
2. Press button on Dynamic Force Control System to release tension in DFS straps (Picture 10).
3. Open both upper and lower Quick Fit Buckle
4. Remove Quick Fit Buckles from keyholes
5. Remove brace from leg

Range of Motion Adjustment (ROM):

The device comes with a 0° extension stop installed. Additional extension stops have been provided 5°, 10°, and 15°. Flexion stop at 90° is also included.

Note: Optimal corrective forces are applied at full extension. If patient requires an extension stop change, make sure to readjust tension in DFS straps if more unloading is required.

To change range of motion:

1. Peel condyle pad from hinge to expose screw.
2. Remove retaining screw.
3. Slide 0° stop out of hinge.
4. Insert desired extension stop.
5. Secure with retaining screw (DO NOT OVER TIGHTEN)
6. For adding flexion stop, follow steps 1,4&5.

Final Inspection: Right leg brace

(Picture 11, 12, 13 & 14).

11. Front
12. Medial side
13. Lateral side
14. Back

Brace Maintenance

Unloader® One pain relief device is designed for minimum maintenance and care. Follow these easy maintenance steps to keep brace in good condition.

Cleaning

- Liner cover, liners, straps and strap pads may be removed for cleaning.
- DO NOT MACHINE WASH.
- DO NOT PLACE IN DRYER
- Hand-wash using mild non-detergent soap.
- Rinse thoroughly.
- Air-dry.
- Strapping Sequence (Picture 15).

If migration occurs, clean liners with an alcohol wipe or damp cloth.

Hinge

- For safety reasons, hinge should only be disassembled by a health care professional.
- Check regularly for foreign materials in hinge i.e. (dirt or grass).
- Clean dirt from hinge using tap water or compressed air.

If brace is used in salt water or chlorinated water, rinse well and air dry brace. A regular service routine will ensure hinge, liners, straps and pads are in optimum working order.

Brace Accessories

Ossur offers accessories that meet specific needs of the Unloader® One. These accessories can be purchased through the brace provider or Ossur Customer Service Departments.

- Liner cover
- Extra Set of Liners
- Replacement Straps
- Strap pads
- Undersleeves are NOT recommended

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