
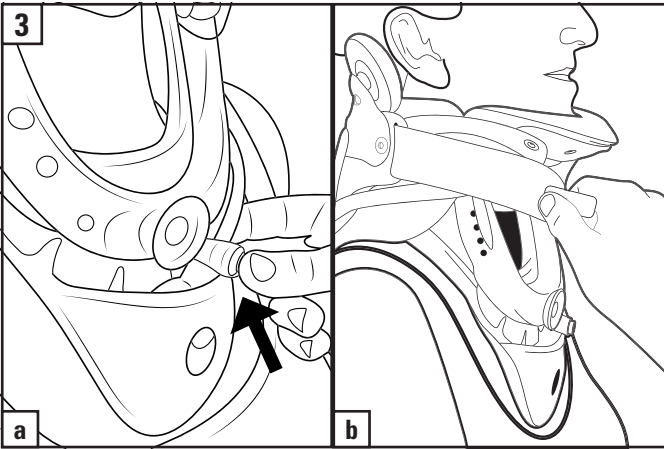
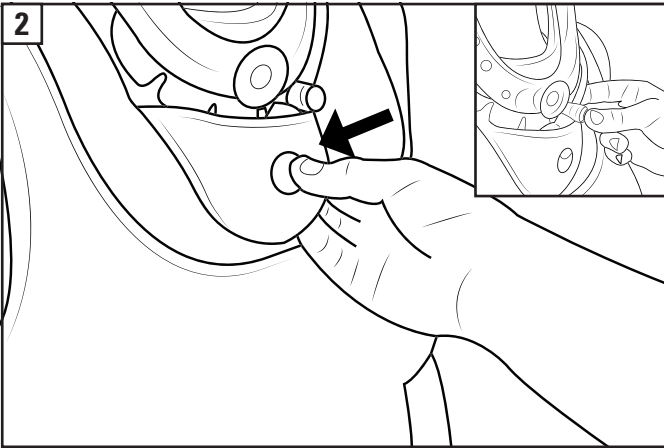
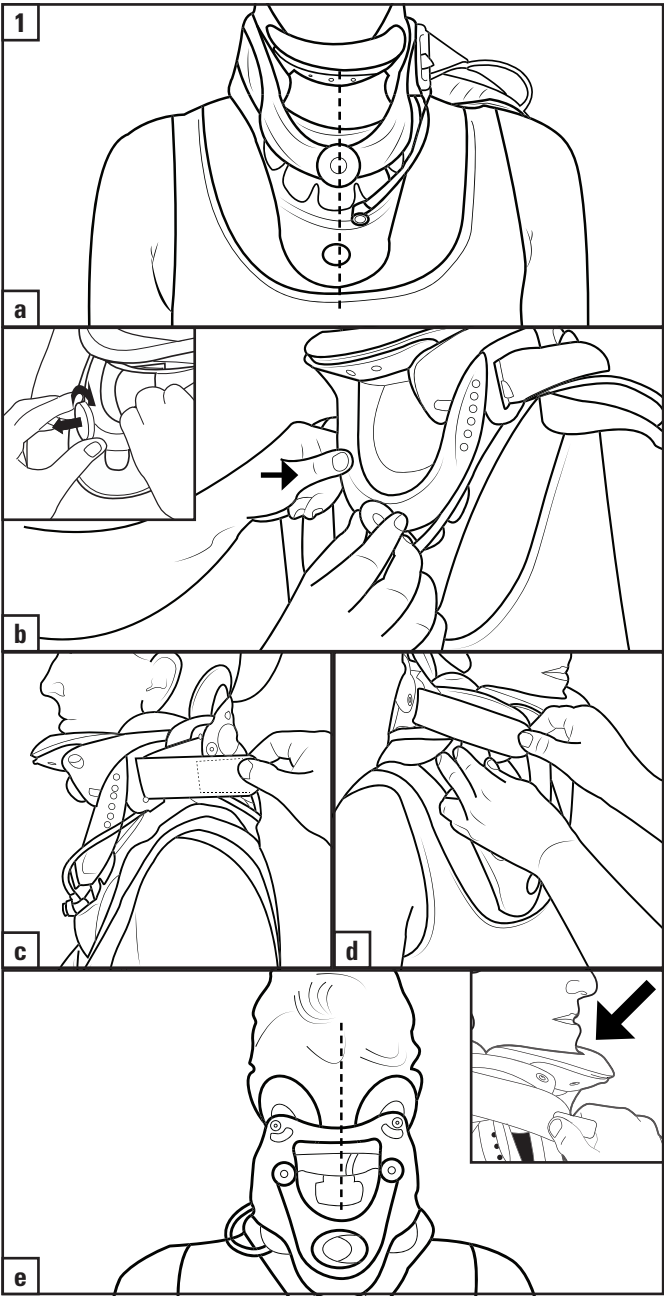
VISTA® MULTIPOST THERAPY COLLAR

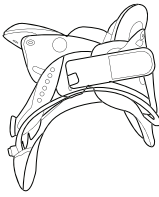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The diagram consists of four panels labeled a, b, c, and d, illustrating the application of a sensor to a patient's neck.

- a.** A hand is shown holding a rectangular sensor. The sensor has two rectangular adhesive pads on its surface. A dashed line indicates the sensor's position relative to the neck.
- b.** The sensor is being applied to the neck. The hand is shown peeling the sensor away from the neck, revealing the adhesive pads.
- c.** The sensor is being adjusted. The hand is shown pressing the sensor onto the neck, ensuring it is properly positioned.
- d.** The sensor is being secured. The hand is shown holding the sensor in place, and a strap is being used to secure it.

LIMITED WARRANTY
Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or parts).

Aspen Medical Products shall have no obligation under this limited warranty in the event:

- a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;
- b) The product is altered;
- c) Any parts not supplied by Aspen Medical Products are inserted into the product; or
- d) The product is not used in accordance with the Aspen Medical Products Instructions for Use.

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
THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING BUT NOT LIMITED TO, ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS IN DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.

This warranty gives you specific legal rights and you may have other rights which vary from state to state.

Warning: To ensure proper use, please review all material.

A listing of authorized distributors in your area is available upon request.

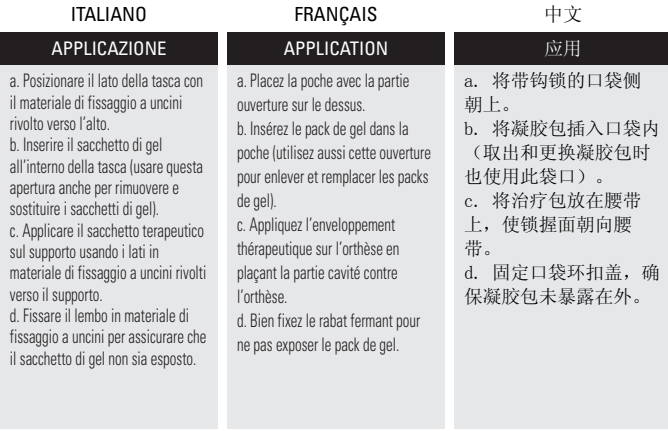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READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE IN ANY MANNER, INCLUDING NEGLIGENCE, BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.